## INFORMACIÓN TÉCNICA

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**For Home Use:** Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild shampoo or dish detergent. Rinse thoroughly after washing. Visually verify that the swivel elbow is not obstructed and allow the mask to air dry.

**For Institutional Use:** Contact the manufacturer for the recommended method of disinfection. When using CIDEX OPA® solution, use at full strength.

#### **TECHNICAL INFORMATION**

Deliverable pressure range: 3 to 20 cm H<sub>2</sub>O  
Dead space: 120 ml  
Pressure drop: 0.31 @ 30 lpm, 1.28 @ 60 lpm

Recommended operating conditions:

Temperature 60°F to 90°F  
Humidity 0°F to 85% RH

Recommended storage conditions:

Temperature 10°F to 90°F  
Humidity 0°F to 50% RH

### **User Instructions for *iQ*® StableFit Headgear**

1. Attach tubing to mask, if not already assembled.
2. Thread upper straps through forehead slots as illustrated, and affix Velcro tabs lightly.
3. Thread lower straps through slots in the mask headgear connector. Thread center bridge strap through top slot on the headgear connector.
4. Place the mask and headgear assembly on head with blue fabric facing outward. Adjust upper and lower headgear straps; unhook the Velcro tabs and pull forward to center the mask over face, and adjust strap tension and length. Affix tabs.
5. Adjust center bridge strap. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

6. Attach mask tubing to device tubing; turn device on and verify airflow through mask.
7. Gently bend pliable ring embedded inside mask shell to change mask shape to create a comfortable fit, and reduce leaks.

#### REMOVING THE MASK AND HEADGEAR

8. Open one of the horizontal straps and detach from headgear connector on the mask.

*iQ*® is a trademark of Sleepnet Corporation. Patent Pending.

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
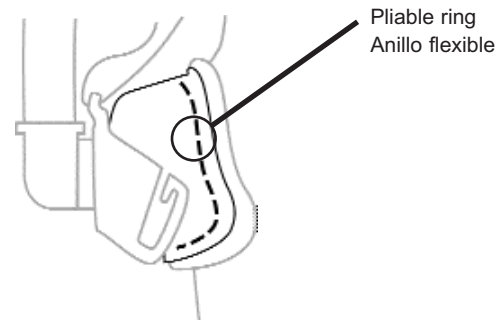
7

## iQ® Ventilation Nasal Mask

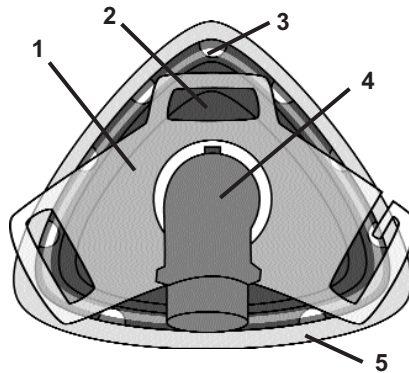CONTENTS: iQ® Ventilation Nasal Mask with iQ StableFit Headgear  
 Rx ONLY DESIGNED AND BUILT IN THE U.S.A.  
 THIS PRODUCT DOES NOT CONTAIN NATURAL RUBBER LATEX

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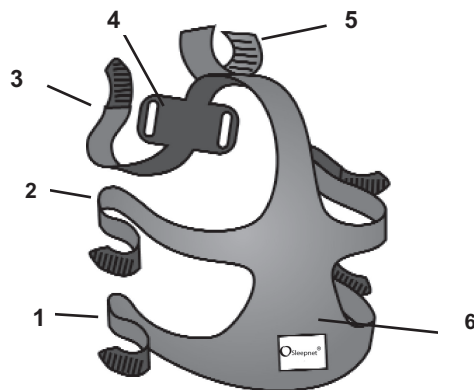
### INTENDED USE

The iQ Ventilation Nasal mask is to be used as an accessory to CPAP/bi-level positive pressure systems that have adequate alarms and safety systems for positive pressure delivery failure. Use of this product is indicated for use with CPAP/BI-LEVEL POSITIVE PRESSURE SYSTEMS CONTAINING EXHALATION VALVES. The iQ Ventilation Nasal Mask is intended for single patient, multi-use in the home environment and multiple patients, multi-use in the hospital/institutional environment. It is intended for use on adult patients > 30 kg.

### GENERAL INFORMATION - WARNINGS AND CAUTIONS

**CAUTION:** U. S. Federal Law restricts this device for sale by or on the order of a physician.

**CAUTION:** Do not expose the iQ® Ventilation Nasal Mask to temperatures above 122°F (50°C).

**WARNING:** This mask should be used with a POSITIVE PRESSURE DEVICE WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. This mask should not be used unless the positive pressure device is turned on and operating properly. When the device is functioning properly, the exhalation valve allows exhaled air to escape to ambient. However, when the device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

**WARNING:** This mask must be used with a device which has ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a device failure. In the event of device failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

**CAUTION:** Consult manufacturer for proper O<sub>2</sub> enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

**WARNING:** If oxygen is used with the mask, the oxygen flow must be turned off when the device is not operating. When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the device enclosure. Oxygen build-up will create a risk of fire.

**WARNING:** No smoking or open flames (such as candles) may be used when oxygen is in use.

**WARNING:** The iQ® Ventilation Nasal Mask must be used with non-conductive tubing.

**WARNING:** If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

### GENERAL INFORMATION

The iQ® Ventilation Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier.

**For Home Use:** Typical expected lifetime of use is 6 to 12 months.

**For Institutional Use:** Due to the nature of high level disinfection, the expected lifetime is typically 15 disinfection cycles.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines. The iQ® Ventilation Nasal Mask and its accessories do not contain natural rubber latex.

NOTE: The iQ Ventilation Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

### CLEANING AND MAINTENANCE

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tears or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ® Ventilation Nasal Mask with care.