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MiniMe® Pediatric Mask

CONTENTS: MiniMe® Pediatric Mask, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The MiniMe® Pediatric Mask is intended to provide an interface for application of CPAP or bi-level therapy. It is intended for patients >2 years old and <12 years old. The MiniMe® Pediatric Mask is intended for single patient, multi-use in the home environment and multiple patients, multi-use in the hospital/institutional environment.

CAUTIONS AND WARNINGS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the MiniMe® Pediatric Mask to temperatures above 122°F (50°C).

CAUTION: Do not use the MiniMe® Pediatric Mask with baseline pressure less than 3 cm H₂O or greater than 20 cm H₂O.

CAUTION: The MiniMe® Pediatric Mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: Do not block or try to seal the vent holes. (If using the non-vented style mask, you must use with a patient circuit that incorporates an exhalation or anti-asphyxia valve as part of the circuit assembly.)

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. Due to the size of the mask, it is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, or by your physician or respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be rebreathed. Rebreathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of positive airway pressure devices and masks.

WARNING: If oxygen is used with this mask, the oxygen flow must be turned off when the CPAP device is not operating. Explanation of the warning: when the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the CPAP device enclosure. Oxygen accumulated in the CPAP device enclosure will create a risk of fire.

WARNING: No smoking or open flames (such as candles) may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

GENERAL INFORMATION

The MiniMe® Pediatric Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. The mask has an expected service life of 6 to 12 months. Please dispose of the mask and accessories in accordance with local and regional guidelines. The MiniMe® Pediatric Mask and its accessories are Latex Free.

NOTE: The MiniMe® Pediatric Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new MiniMe® Pediatric Mask with care.

NOTE: The MiniMe® Pediatric Mask without vent holes can only be used with ventilators having an active exhaust valve.

CLEANING AND MAINTENANCE

For Home Use: Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory or Dove. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

For Institutional Use: USA ONLY: Refer to manufacturer's suggested method of disinfection. When using Cidex OPA® Solution High Level Disinfectant, use at full strength.

NOTE: Inspect mask and vent holes prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures. Vent holes should not be blocked.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.40 cm H₂O; at 100 L/min: 1.20 cm H₂O

A-weighted Sound Pressure Level at 1 m distance: 30.3 db (A); A-weighted Sound Power Level: 38.3 db (A)
DEADSPACE VOLUME (approx.) 21 mL

User Instructions for *MiniMe*® Pediatric Mask Headgear

1. Place headgear on head, with blue fabric facing outward. Hook both straps on the open slots in the headgear connector.
2. Pull straps outward to center the mask over nose. Adjust straps. Headgear sizes Small and Medium feature additional Velcro® tabs.
3. Extend flexible tubing to a comfortable position. Depending on head size, the tubing can be moved upward or downward to assist with placement of mask.

NOTE: The mask should comfortably cover the nose, and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict the nose; child should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

4. Attach mask tubing to CPAP tubing; turn device on and verify airflow through mask.
5. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

REMOVING THE MASK AND HEADGEAR

6. Detach strap from headgear connector.

CE₀₁₂₃ MiniMe® is a trademark of Sleepnet Corporation. Patent Pending.
Visit our website at www.sleep-net.com.

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