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www.sleep-net.com

DECLARATION OF CONFORMITY

MANUFACTURER: SLEEPNET CORPORATION
5 MERRILL INDUSTRIAL DRIVE
HAMPTON, NH USA 03842

AUTHORIZED REPRESENTATIVE: MEDICAL PRODUCT SERVICES GmbH
BORNGASSE 20
35619 BRAUNFELS
GERMANY

PRODUCTS: IQ VENTILATION NASAL MASK, MOJO NON-VENTED
FULL FACE MASK & MINIME PEDIATRIC
VENTILATION MASK

PRODUCT TYPE: POSITIVE PRESSURE BREATHING ATTACHMENT

PRODUCT OPTIONS: HEADGEAR

VARIANTS: N/A

PRODUCT CLASSIFICATION: IIa, ANNEX IX RULE 2

WE HEREWITH DECLARE THAT THE ABOVE MENTIONED PRODUCTS MEET THE PROVISIONS OF
THE COUNCIL DIRECTIVE 93/42/EEC FOR MEDICAL DEVICES (MDD 93/42/EEC), AS AMENDED BY
DIRECTIVE 2007/47/EC.

STANDARDS APPLIED: MDD ESSENTIAL REQUIREMENTS

NOTIFIED BODY: TUV PRODUCT SERVICE, GmbH

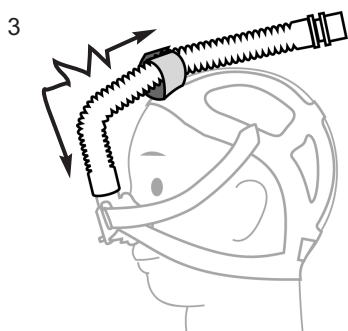
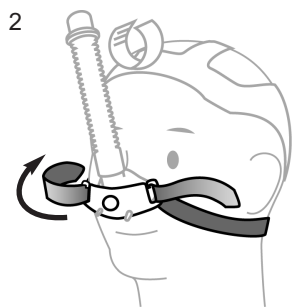
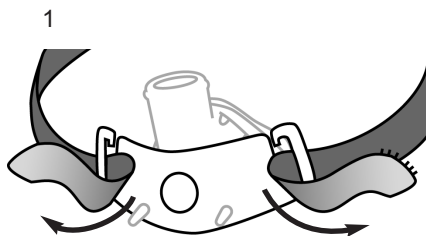
GMDN CODES: NASAL MASK: P36033
FULL FACE MASK: P44521

UMDNS CODE: 12447


THIS DECLARATION APPLIES TO ALL CE MARKED DEVICES MANUFACTURED FROM THE DATE OF
ISSUANCE UNTIL IT IS EITHER SUPERCEDED BY ANOTHER DECLARATION OR WITHDRAWN.

HAMPTON, NH 03842 USA, 2010


Jennifer Kennedy, Regulatory Affairs
Date: 1/12/10

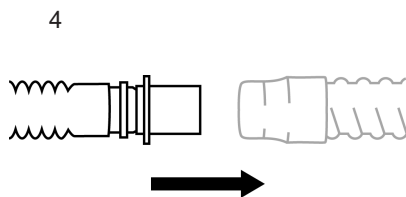
Mask tubing
 Tubo de máscara
 Maskenschlauch
 Tubulure du masque
 Slang masker
 Tubo della maschera
 Σωληνάκι μάσκας
 Maskeslange
 Maskslang
 Tubo da máscara
 Rurka maski
 Maskin letkujärjestelmä
 Maske hortumu

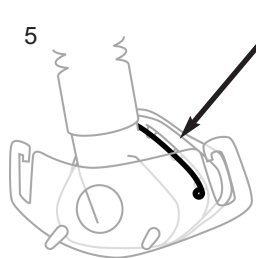Ventilator tubing
 Tubo del respirador
 Ventilatorschlauch
 Tubulure de l'appareil de ventilation
 Ventilatorslang
 Tubo per la ventilazione
 Σωλήνας εξαερισμού
 Ventilatorslange
 Slang från andningsaggregat
 Tubo de ventilador
 Rurka aparatu oddechowego
 Hengityslaitteen letku

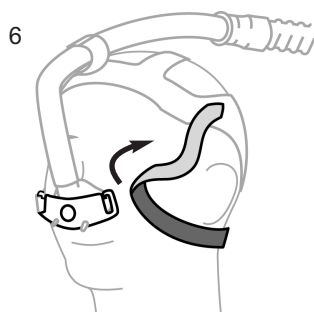Pliable ring
 Anillo flexible
 Biegsamer Ring
 Anneau flexible
 Flexibele ring
 Anello pieghevole
 Εύκαμπτος δακτύλιος
 Bøjelig ring
 Mjuk ring
 Anel flexível
 Miękki pierścień
 Muotoiltava rengas
 Esnek halka

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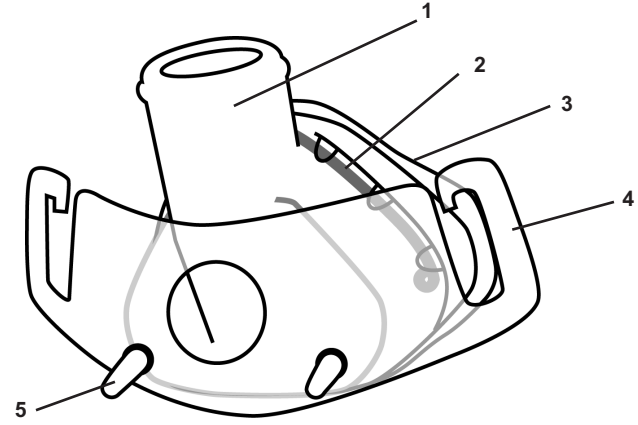
MiniMe® Pediatric Ventilation Mask

CONTENTS: **MiniMe®** Pediatric Ventilation Mask, Tubing Assembly, and Headgear
Rx ONLY LATEX FREE MADE IN U.S.A.

The **MiniMe®** Pediatric Ventilation Mask is intended to provide a patient interface for application of noninvasive ventilation. The mask is to be used as an accessory to ventilators that have adequate alarms and safety systems for ventilator failure, and which are intended to administer positive pressure ventilation for treatment of respiratory failure or respiratory insufficiency. It is intended for patients >2 years old and <12 years old. The **MiniMe®** Pediatric Ventilation Mask is intended for single patient, multi-use in the home environment and multiple patients, multi-use in the hospital/institutional environment.

CAUTIONS AND WARNINGS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the **MiniMe®** Pediatric Ventilation Mask to temperatures above 170°F (75°C).

WARNING: This mask should be used with a POSITIVE PRESSURE VENTILATOR WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the ventilator is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

WARNING: This mask must be used with ventilators which have ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

CAUTION: The **MiniMe®** Pediatric Ventilation Mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. Due to the size of the mask, it is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: If oxygen is used with this mask, the oxygen flow must be turned off when the CPAP device is not operating. Explanation of the warning: when the ventilator is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the ventilator enclosure. Oxygen accumulated in the ventilator enclosure will create a risk of fire.

WARNING: No smoking or open flames (such as candles) may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

GENERAL INFORMATION

The **MiniMe®** Pediatric Ventilation Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. The mask has an expected service life of 6 to 12 months. Please dispose of the mask and accessories in accordance with local and regional guidelines. The **MiniMe®** Pediatric Ventilation Mask and its accessories are Latex Free.

NOTE: The **MiniMe®** Pediatric Ventilation Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new **MiniMe®** Pediatric Ventilation Mask with care.

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CLEANING AND MAINTENANCE

For Home Use: Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory or Dove. Rinse the mask thoroughly after washing. Visually verify that there are no obstructions and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

For Institutional Use: USA ONLY: Refer to manufacturer's suggested method of disinfection. When using Cidex OPA® Solution High Level Disinfectant, use at full strength.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

TECHNICAL INFORMATION DEADSPACE VOLUME (approx.) 21 mL

User Instructions for MiniMe® Pediatric Mask Headgear

1. Place headgear on head, with blue fabric facing outward. Hook both straps on the open slots in the headgear connector.
2. Pull straps outward to center the mask over nose. Adjust straps. Headgear sizes Small and Medium feature additional Velcro® tabs.
3. Extend flexible tubing to a comfortable position. Depending on head size, the tubing can be moved upward or downward to assist with placement of mask.
NOTE: The mask should comfortably cover the nose, and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict the nose; child should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.
4. Attach mask tubing to ventilator tubing; turn device on and verify airflow through mask.
5. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

REMOVING THE MASK AND HEADGEAR

6. Detach strap from headgear connector.

CE₀₁₂₃ MiniMe® is a trademark of Sleepnet Corporation.
Visit our website at www.sleep-net.com.

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