








## Fitting Tips

### The Mojo® Full Face Mask



1. The mask shell can be shaped and re-shaped to follow the contours of the face. If the mask has been shaped previously, return the shell to its original shape by holding the mask with your thumbs inside the shell, parallel, and bend the wire outward until the gel resembles a triangular pear.
2. Turn the mask around (elbow towards you) and rotate the tensioning wheel clockwise as far as it will easily turn; do not screw wheel off its support screw. Bend the forehead pad flat.
3. Assemble the mask to the headgear according to the product instructions. Put mask assembly on head; center mask on face, with top of mask gel cushion on the nose bridge and between the eyes and forehead pad on brow. The mask should rest with the bottom of gel cushion just below the lips and above the chin. Adjust the headgear straps for a comfortable fit.
4. Connect the mask elbow to the airway pressure device tubing and turn the unit on. Begin at a low pressure setting to check mask fit and seal.
5. If you feel air leaks at the bridge of your nose, turn the tensioning wheel counter-clockwise until leaks are reduced. This will divert the force of the upper straps, down the top of the mask shell, and in towards the bridge. (Note: the tensioning wheel can come off the screw if it is turned too far, but can be re-attached.)
6. If you feel air leaks around the cheeks and mouth, gently press on the wire embedded inside the shell to seal the leaks. The wire is very durable and can be maneuvered repeatedly.
7. If necessary, pull the mask away from the face and reset over nose and mouth; move mask higher or lower; readjust headgear. The device tubing can be secured to the headgear frame with the headgear loop strap, draped to either side, or downward.

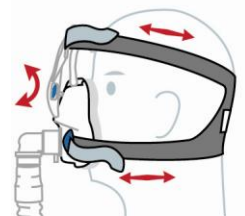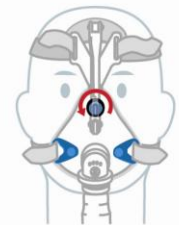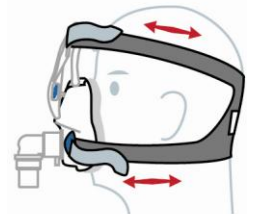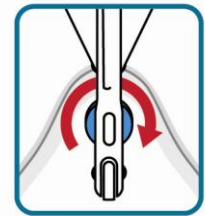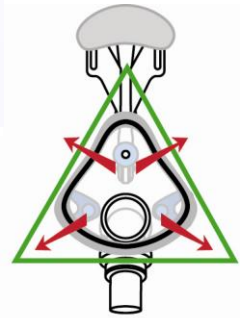


Please contact us if you have any questions about your Mojo Mask or headgear.  
1-800-742-3646 [www.sleepnetmasks.com](http://www.sleepnetmasks.com)

# The MOJO® Full Face Mask

## Trouble Shooting

If the tensioning wheel has unscrewed from its post, it can be re- screwed into place. Hold the mask shell with your thumb inside the shell, pressed against the base of the post; wrap your fingers around the dual beam element, as illustrated, and replace the tensioning screw inside its slot, perpendicular to the post. Turn the screw clockwise until it threads on the post.

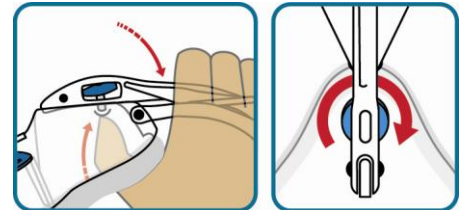

## Mojo Headgear

Mojo Mask, Large, with Mojo Headgear

Mojo Mask, Medium, with Mojo Headgear


Mojo Mask, Small, with Mojo Headgear

#55045

#50837

#50836

#50835

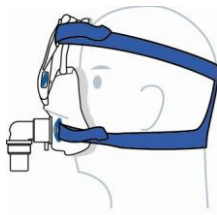

- Lightweight breathable composite headgear
  - Maximizes breathability
  - Adhesive Free
  - Does not contain natural rubber latex
- One size, comfortable and secure

## Mask care

Clean mask daily with mild soap and water; allow to air dry. Store in a cool, dry place away from excessive heat and excessive cold.

DO NOT use excessive and repeated force to bend the pliable ring or nose clip. The wire may lose flexibility and prevent the mask from forming a seal.

DO NOT clean the mask in a washing machine or dish washer.

DO NOT puncture or tear the gel cushion. The gel is non-toxic, but can leak out. Damage to the gel cushion will prevent the mask from forming a seal and will create air leak; if the gel cushion is damaged, the mask must be replaced. This type of damage is not covered by our warranty.

DO NOT try to repair gel cushion with tape or adhesive.

DO NOT block swivel vent holes; this will cause patient to re-breathe exhaled air.

## General Information

- Sizes S, M, L
- 90 day warranty against manufacturer's defects
- Wash mask on a daily basis with mild soap and water
- Mask does not contain natural rubber latex
- Negligible pressure drop throughout mask system
- Compatible with all positive airway pressure devices