Explanation of Symbols  
Explicación de símbolos  
Erklärung der Symbole  
Explication des symboles  
Spiegazione dei simboli

**iQ**® Nasal Mask

with User Instructions for **iQ**® StableFit Headgear

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Part number  
Número de pieza  
Teilenummer  
Référence  
Numero di articolo

Manufacturer  
Fabricante  
Hersteller  
Fabricant  
Fabbrikante

**These instructions contain the following languages:  
English, Spanish, German, French and Italian**

 **Sleepnet**®

**Sleepnet Corporation**

5 Merrill Industrial Drive  
Hampton, NH 03842 USA

Tel: (603) 758-6600

Fax: (603) 758-6699

Toll Free: 1-800-742-3646 (USA AND CANADA ONLY)

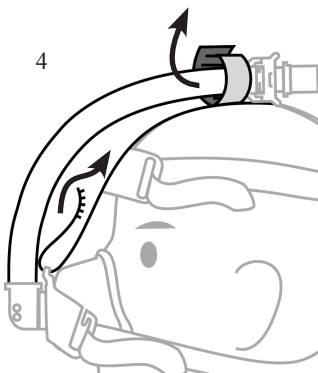
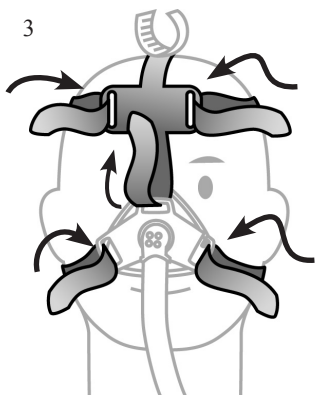
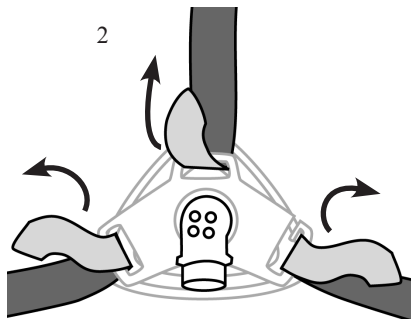
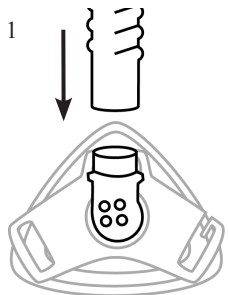
[www.sleepnetmasks.com](http://www.sleepnetmasks.com)

EU Rep:  
Medical Product Services GmbH  
Borngasse 20, 35619 Braunfels Germany  
[info@mps-gmbh.eu](mailto:info@mps-gmbh.eu)

Australian Sponsor:  
Breathing and Medical.  
127 Bowden St, Meadowbank, NSW 2114  
[www.breathingandmedical.com.au](http://www.breathingandmedical.com.au)  
1800 335 333

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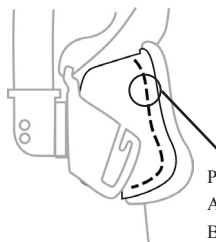
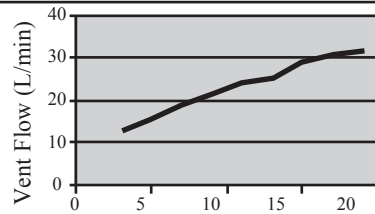
### iQ<sup>®</sup> Nasal Mask

CONTENTS: **iQ<sup>®</sup> Nasal Mask**, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation **iQ<sup>®</sup> Nasal Mask** is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H<sub>2</sub>O for the treatment of adult obstructive sleep apnea.

#### GENERAL INFORMATION - WARNINGS AND CAUTIONS

- CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.
- CAUTION:** Do not expose the **iQ<sup>®</sup> Nasal Mask** to temperatures above 122°F (50°C).
- CAUTION:** This mask may be cleaned and used repeatedly on the same person. Do not sterilize.  
 USA ONLY: For multiple patients use, high level disinfection protocol must be followed.
- WARNING:** At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- WARNING:** This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.
- WARNING:** If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. **WARNING:** No smoking or open flames such as candles may be used when oxygen is in use.
- WARNING:** Do not use the **iQ<sup>®</sup> Nasal Mask** with baseline pressure less than 3 cm H<sub>2</sub>O or greater than 20 cm H<sub>2</sub>O.
- WARNING:** Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

#### GENERAL INFORMATION

The **iQ<sup>®</sup> Nasal Mask** has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The **iQ<sup>®</sup> Nasal Mask** and its accessories are Latex Free.

**NOTE:** The mask has a service life of 6 months.

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#### **CLEANING AND MAINTENANCE - SINGLE PATIENT USE**

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

**NOTE:** Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

**NOTE:** The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ® Nasal Mask with care.

#### **TECHNICAL INFORMATION**

Resistance to air flow at 50 L/min: 1.10 cm H<sub>2</sub>O; at 100 L/min: 2.52 cm H<sub>2</sub>O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

#### **USER INSTRUCTIONS FOR IQ® STABLEFIT™ HEADGEAR**

1. Attach tubing to mask.
2. Thread upper straps through forehead slots as illustrated, and affix Velcro® tabs lightly.
3. Thread lower straps through slots in the mask headgear connector. Thread center bridge strap through top slot on the headgear connector.
4. Place the mask and headgear assembly on head with blue fabric facing outward. Adjust upper and lower headgear straps; unhook the Velcro® tabs and pull forward to center mask over face, and adjust strap tension and length. Affix tabs.
5. Adjust center bridge strap. Attach tubing to headgear with loop strap, if preferred.

**NOTE:** The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

6. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
7. Gently bend pliable ring embedded within mask to create a comfortable fit and reduce leaks.

#### **REMOVING THE MASK AND HEADGEAR**

8. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
9. Open one of the horizontal straps and detach from headgear connector on the mask.

 iQ® is a trademark of Sleepnet Corporation.  
Visit our website at [www.sleepnetmasks.com](http://www.sleepnetmasks.com).

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CE 0123 iQ® es una marca comercial de Sleepnet Corporation.
REZEPTPFLICHTIG LATEXFREI MADE IN U.S.A.