Explanation of Symbols
Explicación de símbolos
Erklärung der Symbole
Explication des symboles
Toelichting bij symbolen
Spiegazione dei simboli
Επεξήγηση των συμβόλων

Forklaring på symboler
Förklaring av symboler
Explicação dos símbolos
Wyjaśnienie symboli
Merkkien selitys
Sembollerin açıklaması

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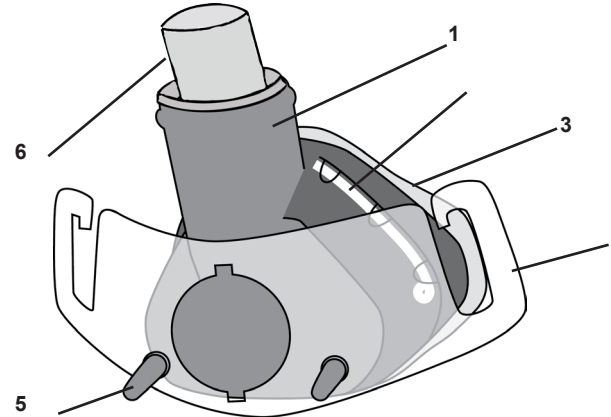
MiniMe® Pediatric Ventilation Mask

CONTENTS: **MiniMe®** Pediatric Ventilation Mask, Tubing Assembly, and Headgear
Rx ONLY MADE IN U.S.A.

DOES NOT CONTAIN NATURAL RUBBER LATEX

The **MiniMe®** Pediatric Ventilation Mask is intended to provide a patient interface for application of noninvasive ventilation. The mask is to be used as an accessory to ventilators that have adequate alarms and safety systems for ventilator failure, and which are intended to administer positive pressure ventilation for treatment of respiratory failure or respiratory insufficiency. It is intended for patients >2 years old and <12 years old. The **MiniMe®** Pediatric Ventilation Mask is intended for single patient, multi-use in the home environment and multiple patients, multi-use in the hospital/institutional environment.

CAUTIONS AND WARNINGS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the **MiniMe®** Pediatric Ventilation Mask to temperatures above 122°F (50°C).

WARNING: This mask should be used with POSITIVE PRESSURE VENTILATOR WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the ventilator is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

WARNING: This mask must be used with ventilators which have ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

CAUTION: The **MiniMe**® Pediatric Ventilation Mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. Due to the size of the mask, it is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: If oxygen is used with this mask, the oxygen flow must be turned off when the CPAP device is not operating. Explanation of the warning: when the ventilator is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the ventilator enclosure. Oxygen accumulated in the ventilator enclosure will create a risk of fire.

WARNING: No smoking or open flames (such as candles) may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

GENERAL INFORMATION

The **MiniMe**® Pediatric Mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com/products/our-promise.cfm>.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: The **MiniMe**® Pediatric Mask and its accessories DO NOT CONTAIN NATURAL RUBBER LATEX.

NOTE: The **MiniMe**® Pediatric Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new **MiniMe**® Pediatric Mask with care.

NOTE: The **MiniMe**® Pediatric Mask without vent holes can only be used with ventilators having an active exhaust valve.

CLEANING AND MAINTENANCE

For Home Use: Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory or Dove. Rinse the mask thoroughly after washing. Visually verify that there are no obstructions and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

For Institutional Use: USA ONLY: Refer to manufacturer's suggested method of disinfection. When using Cidex OPA® Solution High Level Disinfectant, use at full strength.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.38 cm H₂O; at 100 L/min: 1.51 cm H₂O

DEADSPACE VOLUME (approx.) with tubing 57 mL, without tubing 21 mL.

User Instructions for *MiniMe*® Pediatric Mask Headgear

1. Place headgear on head, with blue fabric facing outward. Hook both straps on the open slots in the headgear connector (Figure 1).
2. Pull straps outward to center the mask over nose. Adjust straps. Headgear sizes Small and Medium feature additional Velcro® tabs (Figure 2 and 3).

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NOTE: The mask should comfortably cover the nose, and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict the nose; child should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

3. Attach mask tubing to CPAP tubing; turn device on and verify airflow through mask (Figure 4).
4. If you notice any air leaks, gently press along the Memory Wire™ to seal the leaks (Figure 5).

NOTE: If leaks persist despite customization of Memory Wire and adjustment of headgear straps, try moving the mask higher or lower around the nose to find the best fit. Velcro headgear straps can be readjusted to improve fit as well.

REMOVING THE MASK AND HEADGEAR

5. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing (Figure 6).
6. Open one of the horizontal straps and detach from headgear connector on the mask (Figure 7).

MiniMe® is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com

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CE 0123 **MiniMe®** è un marchio di fabbrica di Sleepnet Corporation.