



## MiniMe® Pediatric Nasal Mask *Fitting Tips*

Once the headgear has been properly attached to the mask, follow the fitting tips below to fit the mask:

1. The MiniMe® Nasal Mask contains Custom Fit Technology™. This soft mask shell can be shaped to follow the contours of a child's face. The Memory Wire™ embedded into the shell locks the customized fit in place. (See Figure A)
1. Detach one Velcro strap by sliding the Velcro loop off the quick release hook, slip the headgear over the child's head and center the mask on the face. (See Figure B).
2. Find the detached Velcro strap and slide the Velcro loop back on the quick release hook.
3. On the large sized headgear, there are 2 sets of Velcro tabs on the headgear straps to allow for the best fit. The unused Velcro tabs can be removed with scissors.
4. For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the Velcro straps, tightening or loosening the straps, and reattaching the Velcro. (See Figure C)
5. The Velcro straps can be attached to the upper or lower portion of the headgear for optimal comfort. (See Figure D)
6. Connect the mask tubing assembly to the child's positive airway pressure device tubing and turn the device on.
7. If you notice any air leaks, gently press along the Memory Wire™ to seal the leaks.
8. If leaks persist despite customization of Memory Wire and adjustment of headgear straps, try moving the mask higher or lower around the nose to find the best fit. Velcro headgear straps can be readjusted to improve fit as well.
9. If you continue to have difficulty obtaining a secure and comfortable fit, consider contacting your local home medical equipment provider for assistance. These companies specialize in helping to fit CPAP and BiPAP masks.

Figure A



Figure B

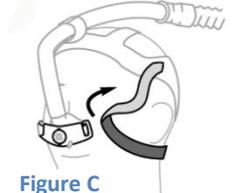


Figure C



Figure D

