Explanation of Symbols Which Appear on the Device Labeling  
Explicación de símbolos que aparecen en la etiqueta del dispositivo  
Erklärung der Symbole auf dem Etikett dieses Gerätes  
Explication des symboles qui apparaissent sur l'étiquette de l'appareil  
Spiegazione dei simboli che figurano nelle etichette del dispositivo  
Пояснение символов, используемых для маркировки устройства

Caution or Warning  
Precaución o advertencia  
Achtung oder Warnung  
Avertissement ou mise en garde  
Attenzione o avvertenza  
Внимание или Осторожно

Refer to Instructions for Use  
Consulte las instrucciones de uso.  
Siehe Gebrauchsanleitung  
Se reporter au Mode d'emploi  
Consultare le Istruzioni per l'uso  
См. инструкции по применению

CE Certification Mark  
Marca de certificación CE  
CE Zertifizierungsmarke  
Marque CE  
Marcatura CE  
Знак сертификации CE

Lot number  
Número de lote  
Losnummer  
Numéro du lot  
Numero di lotto  
Номер партии

Manufacturer  
Fabricante  
Hersteller  
Fabricant  
Fabbricante  
Изготовитель

This Product is not manufactured with Natural Rubber Latex.  
Este producto no se fabrica con látex de caucho.  
Dieses Produkt ist nicht mit Kautschuklatex hergestellt.  
Ce produit est fabriqué sans latex.  
Il prodotto non contiene lattice.  
Настоящий продукт изготавливается без применения натурального каучукового латекса.

Part number  
Número de pieza  
Teilenummer  
Référence  
Numero di articolo  
Номер по каталогу

Use by Date  
Fecha de caducidad  
Verfallsdatum  
Date limite d'utilisation  
Da utilizzare entro la data indicata  
Использовать до указанной даты

Date of Manufacture  
Fecha de fabricación  
Herstellungsdatum  
Date de fabrication  
Data di produzione  
Дата изготовления

Medical Device  
Producto sanitario  
Medizinisches Gerät  
Dispositif médical  
Dispositivo medico  
Медицинское устройство

EU Rep:  
Medical Product Services GmbH  
Borngasse 20, 35619 Braunfels Germany  
info@mps-gmbh.eu

Australian Sponsor:  
Breathing and Medical.  
127 Bowden St, Meadowbank, NSW 2114  
www.breathingandmedical.com.au  
1800 335 333

## *iQ*® Nasal Mask with User Instructions for *iQ*® 3 Point Headgear

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



These instructions contain the following languages:  
English, Spanish, German, French, Italian, and Russian

 Sleepnet®


**Sleepnet Corporation**  
5 Merrill Industrial Drive  
Hampton, NH 03842 USA  
Tel: (603) 758-6600  
Fax: (603) 758-6699  
Toll Free: 1-800-742-3646 (USA ONLY)  
www.sleepnetmasks.com

90112 Rev. L. 2020-12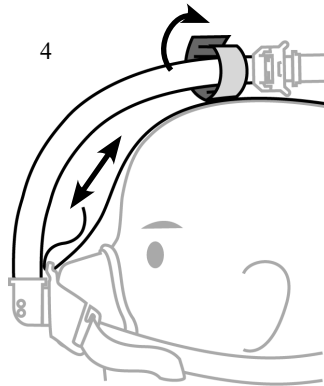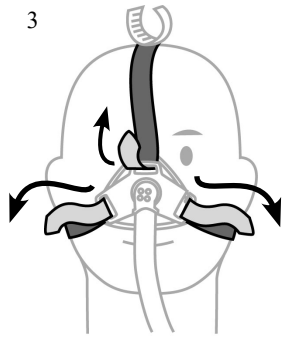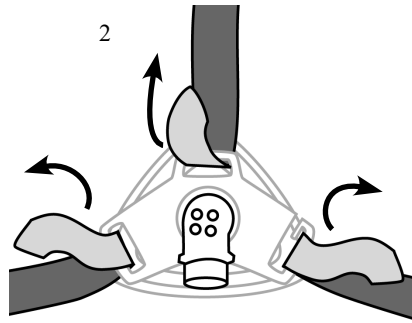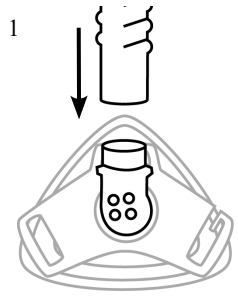

This page intentionally left blank.

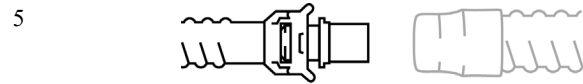Mask tubing  
 Tubo de máscara  
 Maskenschlauch  
 Tubulure du masque  
 Tubo della maschera  
 Трубка маски

CPAP tubing  
 Tubo de CPAP  
 CPAP-Schlauch  
 Tubulure PPE  
 Tubo del dispositivo CPAP  
 Трубка СИПАП

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## **iQ**® Nasal Mask

CONTENTS: **iQ**® Nasal Mask, Tubing Assembly, and Headgear  
Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation **iQ**® Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H<sub>2</sub>O for the treatment of adult obstructive sleep apnea.

## GENERAL INFORMATION - WARNINGS AND CAUTIONS

**CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.

**CAUTION:** Do not expose the **iQ**® Nasal Mask to temperatures above 122°F (50°C).

**CAUTION:** This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

**WARNING:** At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

**WARNING:** This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

**WARNING:** If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

**WARNING:** No smoking or open flames such as candles may be used when oxygen is in use.

**WARNING:** Do not use the **iQ**® Nasal Mask with baseline pressure less than 3 cm H<sub>2</sub>O or greater than 20 cm H<sub>2</sub>O.

**WARNING:** Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

## GENERAL INFORMATION

The **iQ**® Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The **iQ**® Nasal Mask and its accessories are Latex Free. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

**NOTE:** The mask has a service life of 6 months.

#### CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

**NOTE:** Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

**NOTE:** The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new **iQ** Nasal Mask with care.

#### TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cm H<sub>2</sub>O; at 100 L/min: 2.52 cm H<sub>2</sub>O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

#### USER INSTRUCTIONS FOR **iQ** 3POINT HEADGEAR

1. Attach tubing to mask.
2. Thread lower straps through slots in the mask headgear connector, and thread center strap through top slot on headgear connector.
3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.
4. Adjust center strap. Attach tubing to headgear with loop strap, if preferred.

**NOTE:** The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
6. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

#### REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
8. Open one of the horizontal straps and detach from headgear connector on the mask.

 **iQ** is a trademark of Sleepnet Corporation.

Visit our website at [www.sleepnetmasks.com](http://www.sleepnetmasks.com).

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**iQ**® ist ein Markenzeichen der Sleepnet Corporation.