



Ascend[®] Nasal

replaceable AIR[°]gel cushion masks

featuring Advanced Cushion Technology

Rise to a higher level of comfort and convenience with The Ascend nasal mask: our first-ever nasal mask featuring a replaceable cushion. Our patented AIR[°]gel replaceable cushion is easily detached and reattached for easy cleaning and cushion replacement, saving both time and money.

Ascend to a Higher Level of Comfort and Convenience

Flexible Spacebar[™]

Unique bendable design provides superior stability, holding a seal without contacting the skin

AIR[°]gel

Maximum comfort, minimum weight. Our patented AIR[°]gel minimizes pressure points and skin irritation.

Replaceable Cushion

Now you can replace the cushion without having to replace the whole mask. Available in three sizes.

S M L

Advanced Cushion Technology[™] (A.C.T.)

Our ergonomic cushion technology delivers an effective seal for the most comfortable fit.

Quick-Release Headgear


Lightweight, breathable headgear slides off easily with one hand.

360° Rotating Elbow

For optimal tube positioning and easier mobility

Active Headgear Connector[™]

Moves as the patient moves for a reliable seal.

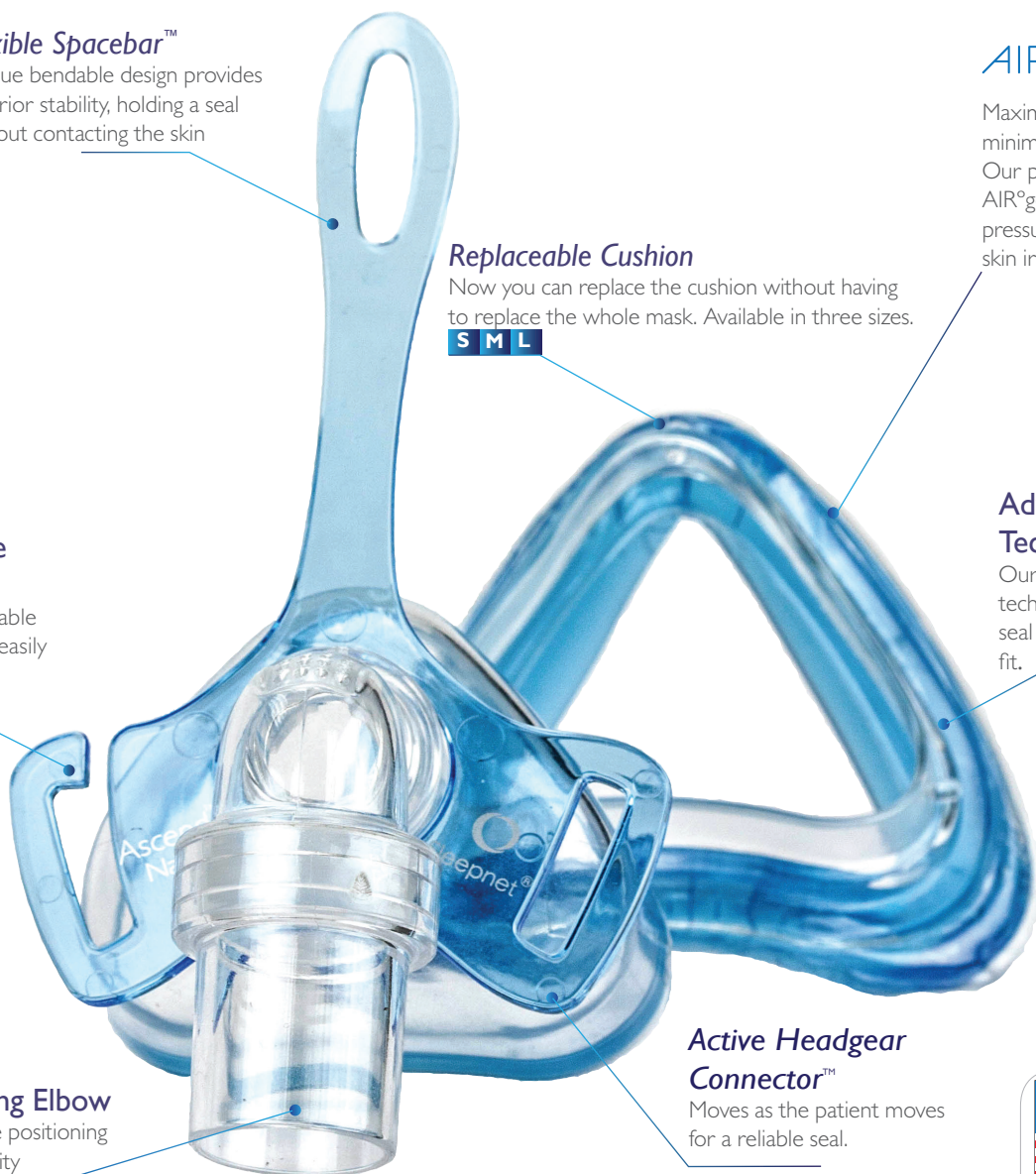

Made in the USA
of US and imported parts

What makes our Ascend Nasal Mask different?

- Our easy-to-use replaceable cushion allows you to extend the life of your mask.
- *Flexible Spacebar* is designed to provide superior stability without contacting the skin.
- *Active Headgear Connector* moves when the patient moves to help maintain an effective seal.
- Available in three sizes with simple and breathable quick-release EZFit headgear.

Advanced Cushion Technology (A.C.T.)

Our patented AIR°gel with an ergonomic design and luxurious feel reduces pressure while maintaining an effective seal.



Easy-to-Replace Cushion



Disassembly: Just squeeze the sides of the cushion in to release and snap in to replace.



Assembly: Place the top alignment pin in the notch and push the bottom ones in until the pins fit securely.

Ascend Nasal Masks with EZFit Disposable Headgear Specifications:

Product Number	Description	Leak Port	Cushion Size	Mask Weight with cushion	Mask Dimensions with cushion
50174	Mask System & headgear	✓	S, M, & L	NA	NA
50179	Mask System with attached headgear	✓	S, M, & L	NA	NA
50187	Mask & headgear	✓	Small	57.9g	3.70"D x 2.75"W x 5.40"H
50188	Mask & headgear	✓	Medium	58.9g	3.80"D x 2.95"W x 5.40"H
50189	Mask & headgear	✓	Large	63.0g	3.97"D x 3.30"W x 5.40"H
50157	Mask with attached headgear	✓	Small	57.9g	3.70"D x 2.75"W x 5.40"H
50158	Mask with attached headgear	✓	Medium	58.9g	3.80"D x 2.95"W x 5.40"H
50159	Mask with attached headgear	✓	Large	63.0g	3.97"D x 3.30"W x 5.40"H
50201	AIR°gel Cushion	✓	Small	57.9g	3.70"D x 2.75"W x 5.40"H
50202	AIR°gel Cushion	✓	Medium	58.9g	3.80"D x 2.95"W x 5.40"H
50203	AIR°gel Cushion	✓	Large	63.0g	3.97"D x 3.30"W x 5.40"H
55051	EZFit Headgear	NA	NA	NA	NA

Ascend Full Face
replaceable AIR°gel cushion masks

Also available as a
Full Face Mask



Contact us today.

To place your order, call an authorized distributor.
Call 1.800.742.3646
Visit www.sleepnetmasks.com to learn more.
Sleepnet Corporation, 5 Merrill Industrial Drive,
Hampton, New Hampshire 03842 USA



Does not contain
natural rubber latex

