**Instrucciones de uso del casco StableFit *iQ* para la máscara nasal *iQ*
iQ Nasenmaske mit Gebrauchsanleitung für die *iQ* StableFit-
Kopfgurtvorrichtung
Masque nasal *iQ* avec mode d'emploi pour casque StableFit *iQ*
Maschera nasale *iQ* con Istruzioni per l'uso della cuffia *iQ* StableFit
Назальная маска *iQ* с инструкцией пользователя для системы
фиксации на голову *iQ* StableFit**

Caution or Warning
Precaución o advertencia
Achtung oder Warnung
Avertissement ou mise en garde
Attenzione o avvertenza
Внимание или Осторожно

Lot number
Número de lote
Losnummer
Numéro du lot
Numero di lotto
Номер партии

Refer to Instructions for Use
Consulte las instrucciones de uso.
Siehe Gebrauchsanleitung
Se reporter au Mode d'emploi
Consultare le Istruzioni per l'uso
См. инструкции по применению

Use by Date
Fecha de caducidad
Verfallsdatum
Date limite d'utilisation
Da utilizzare entro la data indicata
Использовать до указанной даты

Does not contain natural rubber latex
No contiene látex de caucho natural
Enthält keinen Naturkautschuklatex
Ne contient pas de latex en caoutchouc naturel
Non contiene lattice di gomma naturale
Не содержит натурального каучукового латекса

Medical Device
Producto sanitario
Medizinisches Gerät
Dispositif médical
Dispositivo medico
Медицинское устройство

CE Certification Mark
Marca de certificación CE
CE Zertifizierungsmarke
Marque CE
Marcatura CE
Знак сертификации CE

Date of Manufacture
Fecha de fabricación
Herstellungsdatum
Date de fabrication
Data di produzione
Дата изготовления

Part number
Número de pieza
Teilenummer
Référence
Numero di articolo
Номер по каталогу

Manufacturer
Fabricante
Hersteller
Fabricant
Fabbicante
Изготовитель

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iQ[®] Nasal Mask

CONTENTS: iQ[®] Nasal Mask, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation iQ[®] Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H₂O for the treatment of adult obstructive sleep apnea.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the iQ[®] Nasal Mask to temperatures above 122°F (50°C).

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. **WARNING:**

No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Do not use the iQ[®] Nasal Mask with baseline pressure less than 3 cm H₂O or greater than 20 cm H₂O.

WARNING: Consult manufacturer for proper O₂ enrichment tubing.

GENERAL INFORMATION

The iQ[®] Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The iQ[®] Nasal Mask and its accessories are Latex Free. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

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CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ® Nasal Mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cm H₂O; at 100 L/min: 2.52 cm H₂O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

USER INSTRUCTIONS FOR iQ® STABLEFIT™ HEADGEAR

1. Attach tubing to mask.
2. Thread upper straps through forehead slots as illustrated, and affix Velcro® tabs lightly.
3. Thread lower straps through slots in the mask headgear connector. Thread center bridge strap through top slot on the headgear connector.
4. Place the mask and headgear assembly on head with blue fabric facing outward. Adjust upper and lower headgear straps; unhook the Velcro® tabs and pull forward to center mask over face, and adjust strap tension and length. Affix tabs.
5. Adjust center bridge strap. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

6. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
7. Gently bend pliable ring embedded within mask to create a comfortable fit and reduce leaks.

REMOVING THE MASK AND HEADGEAR

8. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
9. Open one of the horizontal straps and detach from headgear connector on the mask.

iQ® is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com.

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