Manufacturer	Výrobce	Tootja
Fabricante	Κατασκευαστής	Ražotājs
Hersteller	Gyártó	Gamintojas
Fabricant	Prodúctor	Producēt
Fabbricante	Výrobca	Изготовитель
Fabrikant	Üretici	Proizvodac
Valmistaja	Fabricante	Izdeļotājs
製造商	Tillverkare	
製造元	Fabrikant	
Производитель	الجهة المصنعة	
Proizvodač	Producēt	

Date of Manufacture	Datum výroby	Tootmiskuupeäev
Fecha de fabricación	Ημερομηνία κατασκευής	Izgatavošanas datums
Herstellungsdatum	A gyártás időpontja	Pagamimino data
Date de fabrication	Data producerii	Data producției
Data di produzione	Dátum výroby	Дата изготовления
Vervaardigingsdatum	Üretim Tarihi	Datum proizvodnje
Valmistuspäivämäärä	Data de fabrico	Datum izdeļave
製造日期	Tillverkningsdatum	
製造年月日	Fabrikācijasdato	
Дата на производство	تاريخ التصنيع	
Datum proizvodnje	Fremstillingsdato	

Medical Device	Orvostechikai eszköz	Медицинское устройство
Producto sanitario	Dispozitiv medical	Medicinski uređaj
Medizinisches Gerät	Zdravotnicka pomůcka	Medicinski pripomoček
Dispositif médical	Medikal Cihaz	
Dispositivo medico	Dispositivo médico	
Medisch hulpmiddel	Medicinsk utrustning	
Lääkinnällinen laite	Medisinsk enhet	جهاز طبي
医療器械	Medicinsk enhed	
Медицинское изделие	Medicinsisade	
Medicinski uređaj	Medicinska ierīce	
Zdravotnický prostředek	Medicinos prietaisai	
Ιατρική Συσκευή	Urządzenie medyczne	

Use by Date	Χρησιμοποιήστε μέχρι την	“Izlietot līdz” datums
Fecha de caducidad	Ημερομηνία	Galojimo data
Verfallsdatum	Felhasználhatósági idő	Data przydatności do użycia
Date limite d'utilisation	A se folosi până la data	Использовать до указанной даты
Da utilizzare entro la data indicata	Použite do dátumu	Datum upotrebe
Uiterste houdbaarheidsdatum	Son Kulanma Tarihi	Datum uporabnosti
Viimeinen käyttöpäivä	Prazo de validade	
此日期前使用	Bör användas före	
使用期限	Brukes innen	’سخدم قبل التاريخ’
Срок на годност		
Upotrijebiti do naznačenog datuma	Holdbarhedsdato	
Datum pouzitelności	Tarvitada enne kõlblikustähtpäeva	

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Australian Sponsor:
Breathing and Medical.
127 Bowden St, Meadowbank. NSW 2114
www.breathingandmedical.com.au
1800 335 333

EU Rep:
Medical Product Services GmbH
Borngasse 20, 35619 Braunfels Germany
info@mps-gmbh.eu

MiniMe® 2 Nasal Non-Vented Mask

These instructions contain the following languages:

English, Spanish, German, French, Italian, Japanese, Dutch, Finnish, Chinese,
Bulgarian, Croatian, Czech, Greek, Hungarian, Romanian, Slovakian, Turkish,
Portuguese, Swedish, Norwegian, Arabic, Danish, Estonian, Latvian, Lithuanian,
Polish, Russian, Serbian, and Slovenian

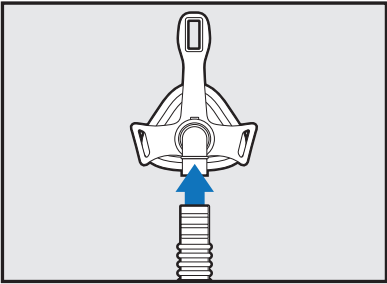
Sleepnet Corporation
5 Merrill Industrial Drive
Hampton, NH 03842 USA
Tel: (603) 758-6600
Fax: (603) 758-6699
Toll Free: 1-800-742-3646 (USA)
www.sleepnetmasks.com

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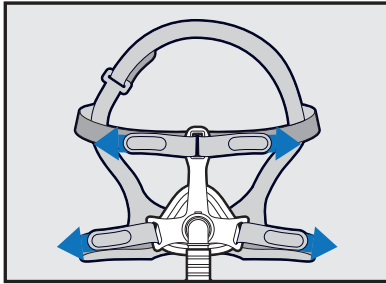
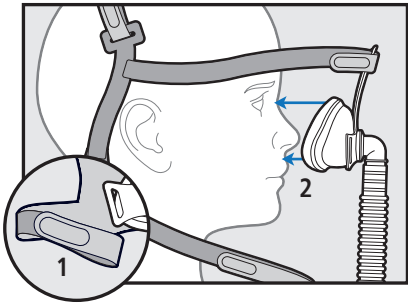
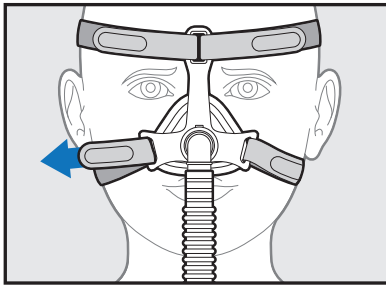
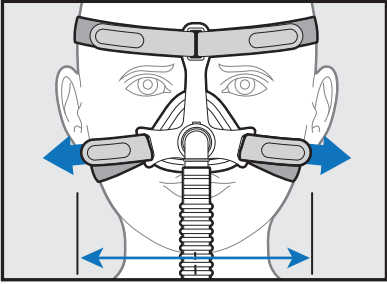
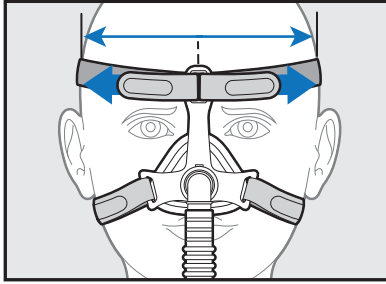
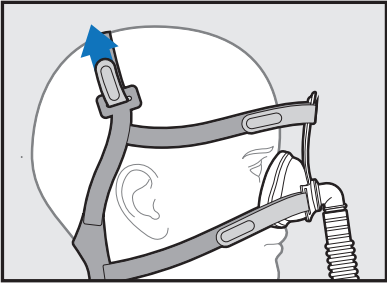
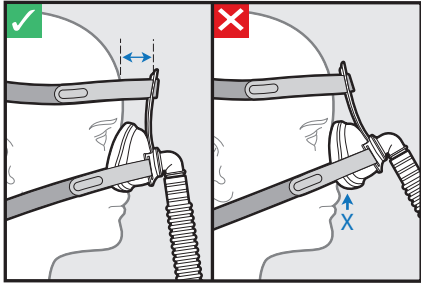
MiniMe® 2 Nasal Non-Vented Mask

CONTENTS: MiniMe® 2 Pediatric Nasal Non-Vented Mask, Tubing, and Headgear
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX
Rx ONLY MADE IN U.S.A.

The MiniMe 2 Pediatric Nasal Non-Vented Mask is intended to provide an interface for application of positive pressure therapy. The mask is to be used as an accessory to CPAP or Bi-level positive pressure systems CONTAINING EXHALATION VALVES. It is intended for patients >2 years old and <12 years old. It is intended for single patient, multi-use in the home, hospital, or institutional environment.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the mask to temperatures above 122°F (50°C).

WARNING: Do not use the Mask with baseline pressure less than 4 cm H₂O or greater than 20 cm H₂O.

CAUTION: The mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: This mask is not suitable for providing life support ventilation.

WARNING: The mask does not have exhalation holes and must be used with a patient circuit that incorporates an exhalation valve.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. The mask is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The vent holes in the patient circuit should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, air from the device flushes the exhaled air out through the vent holes in the patient circuit. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

WARNING: Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

WARNING: Significant unintentional leak may occur without properly fitting the mask. Follow the USER INSTRUCTIONS FOR MINIME 2 NASAL MASK provided.

WARNING: If the mask is prescribed for children with serious congenital conditions or cognitive impairments then the positive pressure device must have the appropriate alarms. The device, mask, and alarms must be tested before starting therapy.

GENERAL INFORMATION

The mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit www.sleepnetmasks.com. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX

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Upor pri pretoku zraka 50 l/min: 1,10 cmH₂O

Upor pri pretoku zraka 100 l/min: 4,20 cmH₂O

Mrtvi prostor (pribl.): velikost **XS** – 20 ml, velikost **S** – 25 ml, velikost **M** – 37 ml, velikost **L** – 40 ml, cevje – 35 ml

Delovna temperatura: Maske ne izpostavljajte temperaturam, višjim od 50 °C (122 °F).

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NOTE: The mask without vent holes can only be used with positive pressure devices that have an exhaust valve.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Hand wash the mask and components in warm water using a mild detergent. Rinse thoroughly after washing. Visually verify that the swivel elbow is not obstructed and allow the mask to air dry.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cmH₂O

Resistance to air flow at 100 L/min: 4.20 cmH₂O

Dead space (approx): **XS size-** 20 ml, **S size-** 25 ml, **M size-** 37 ml, **L size-** 40 ml, **Tubing-** 35 ml

Operating Temperature: Do not expose the mask to temperatures above 122°F (50°C)

USER INSTRUCTIONS FOR THE MINIME 2 NASAL MASK

1. Attach the tubing assembly to the MiniMe 2 Mask (Figure 1).
2. Lay headgear flat with light gray side facing up with the head strap at the top as shown in Figure 2. Place mask gel side down in the center of the headgear and thread all 4 straps through the slots as shown (Figure 2).
3. To place mask on child, detach one or both lower straps from the mask. Slip the headgear over the head. Seat the bottom cushion of the mask between the upper lip and nose, and tip the mask up to contact the nose (Figure 3).
NOTE: The Touchless Spacebar™ should not contact the forehead.
4. Find the detached hook and loop fastener strap and reattach (Figure 4).
5. Ensure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head.
6. Adjust the bottom headgear straps by using the hook and loop fastener tabs (Figure 5).
7. Adjust the upper headgear straps by using the hook and loop fastener tabs: detach the hook and loop fastener tabs then pull straps forward to center the mask over face. Ensure headgear is centered by aligning the two straps. Then pull straps back, adjust strap tension, and affix tabs (Figure 6).
8. Adjust the head strap by using the hook and loop fastener tab (Figure 7).
9. If mask is uncomfortable repeat steps 6-9.
NOTE: Do NOT overtighten the headgear.
NOTE: The Touchless Spacebar™ should NOT touch the forehead. It should be parallel to the forehead (Figure 8).
NOTE: For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the straps, tightening or loosening the straps, and reattaching the hook and loop fastener tabs.
10. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 9).
11. Gently press the edges of the mask to change mask shape, create a comfortable fit, and reduce leaks.
NOTE: The MiniMe 2 Nasal Mask contains Custom Fit Technology™. This soft mask shell can be shaped to follow the contours of a child's face. The Memory Wire™ embedded into the shell locks the customized fit in place.
12. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
13. If leaks are noticed at the bridge of the nose, slightly tighten the top straps as described in in step 7.
The Touchless Spacebar™ should not contact the forehead.
14. If leaks are noticed at the upper lip or below the nose, slightly tighten the bottom straps

MiniMe® 2 is a trademark of Sleepnet Corporation.
Visit our website at www.sleepnetmasks.com.

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