Explanation of Symbols Which Appear on the Device Labeling
Explicación de símbolos que aparecen en la etiqueta del dispositivo
Erklärung der Symbole auf dem Etikett dieses Gerätes
Explication des symboles qui apparaissent sur l'étiquette de l'appareil
Spiegazione dei simboli che figurano nelle etichette del dispositivo

Caution or Warning
Precaución o advertencia
Achtung oder Warnung
Avertissement ou mise en garde
Attenzione o avvertenza

Refer to Instructions for Use
Consulte las instrucciones de uso.
Siehe Gebrauchsanleitung
Se reporter au Mode d'emploi
Consultare le Istruzioni per l'uso

CE Certification Mark
Marca de certificación CE
CE Zertifizierungsmarke
Marque CE
Marcatura CE

Medical Device
Producto sanitario
Medizinisches Gerät
Dispositif médical
Dispositivo medico

Manufacturer
Fabricante
Hersteller
Fabricant
Fabbricante

Use by Date
Fecha de caducidad
Verfallsdatum
Date limite d'utilisation
Da utilizzare entro la data indicata

Lot number
Número de lote
Losnummer
Numéro du lot
Numero di lotto


This Product is not manufactured with Natural Rubber Latex.
Este producto no se fabrica con látex de caucho.
Dieses Produkt ist nicht mit Kautschuklatex hergestellt.
Ce produit est fabriqué sans latex.
Il prodotto non contiene lattice.

Part number
Número de pieza
Teilenummer
Référence
Numero di articolo

Date of Manufacture
Fecha de fabricación
Herstellungsdatum
Date de fabrication
Data di produzione

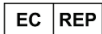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Innova® Nasal Non-Vented Mask

CONTENTS: Innova® Nasal Non-Vented Mask with headgear
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX

Rx ONLY MADE IN U.S.A.

The Innova® Nasal Non-Vented Mask is intended to be used as an accessory to CPAP/bi-level positive pressure systems containing exhalation valves. This product is indicated for use with CPAP/BI-LEVEL systems that have adequate alarms for positive pressure delivery failure. The mask is intended for single patient, multi-use in the home, hospital or institutional environment on adult patients (>30kg/66 lbs).

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the mask to temperatures above 140°F (60°C).

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

CAUTION: As with all positive airway pressure systems, significant leak may occur between the mask and patient's face. The leak may vary depending on the patient's position and sleep stage. Variation in leak may significantly affect patient ventilation, potentially causing profound hypoventilation or hyperventilation. The magnitude of this effect depends on the positive pressure device's ability to compensate for leak.

WARNING: This mask should not be used if the patient cannot respond to potential positive pressure device failure by breathing through their mouth.

WARNING: This mask should be used with a POSITIVE PRESSURE DEVICE WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure device is turned on and operating properly. When the device is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

WARNING: This mask must be used with devices which have ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a device failure. In the event of device failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

WARNING: If oxygen is used with this mask, the oxygen flow must be turned off when the positive pressure device is not operating. Explanation of the warning: When the positive pressure device is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

CAUTION: When used in combination with other medical devices not intended to be combined with the mask, safety can be decreased or performance can be altered (e.g. in combination with a humidifier for medical use, nebulizer, heat and moisture exchanger (HME), filters, bi-level positive airway pressure equipment, self-adjusting equipment, or additional oxygen supply or any exhaust port).

WARNING: Do not use the mask with baseline pressure less than 4 cm H₂O or greater than 20 cm H₂O.

CAUTION: Consult manufacturer for proper O₂ enrichment tubing.

WARNING: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

WARNING: Significant unintentional leak may occur without properly fitting the mask. Follow the user instructions provided below.

WARNING: Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the Innova® Nasal Non Vented Mask and consequently the safety or the quality of the therapy.

GENERAL INFORMATION

The mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com>. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX

NOTE: The mask without vent holes can only be used with positive pressure devices that have an exhaust valve.

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CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory®. Rinse thoroughly after washing. Visually verify that the swivel elbow is not obstructed and allow the mask to air dry.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new mask with care.

TECHNICAL INFORMATION

Therapy pressure range: 4-20 cmH₂O

Resistance to air flow at 30 L/min: 0.10 cmH₂O

Resistance to air flow at 50 L/min: 0.33 cmH₂O

Resistance to air flow at 60 L/min: 0.46 cmH₂O

Resistance to air flow at 100 L/min: 1.42 cmH₂O

Dead space (approx): **S/M size** - 78 ml **M/L size** - 100 ml

Operating Temperature: Do not expose the mask to temperatures above 140°F (60°C)

USER INSTRUCTIONS FOR THE INNOVA® NASAL MASK

1. Remove the mask and the headgear from the package.
 2. Place the headgear on a table, label side down with the narrowest part close to you.
 3. Set the mask with the gel side down onto the center of the headgear.
 4. Take each headgear strap and connect it to its respective location on the mask by looping the strap through the headgear slots and attaching it back to itself (Figure 1).
 5. Once the headgear has been properly attached to the mask, follow the fitting tips below to fit the mask:
 6. Detach one headgear strap by sliding the loop off the quick release hook (Figure 2), slip the headgear over the head and center the mask on the face.
 7. Find the detached headgear strap and slide the loop back on the quick release hook (Figure 2).
 8. Be sure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head (Figure 3).
 9. Adjust the bottom headgear straps by using the hook and loop tabs. Make sure the Active Headgear Connector™ is parallel to the face and that there is equal tension from the headgear on both sides. After tightening, make sure the Active Headgear Connector™ can move freely (Figure 4).
- NOTE:** Do NOT over tighten the headgear.
- NOTE:** The Touchless Spacebar™ will be tilted away from the face after tightening the bottom straps.
10. Adjust the upper headgear straps by using the tabs. Be sure they are tightened evenly and that the Touchless Spacebar™ is parallel to the forehead (Figure 5).
- NOTE:** The Touchless Spacebar™ should NOT touch the forehead. It should be parallel to the forehead (Figure 6).
11. Adjust the bottom headgear straps to make sure the bottom of the mask is sealed comfortably on the face.
 12. For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the hook and loop tabs, tightening or loosening the straps, and reattaching the tabs.
 13. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 7).
 14. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
 15. In the event of leaks, tighten the hook and loop tabs closest to the area of leak. Do NOT over tighten the headgear.

Innova® is a registered trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com.

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Innova® es una marca comercial de Sleepnet Corporation.
REZEPTPFLICHTIG GEBILDET IN U.S.A.