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## Innova® Hospital Nasal Non-Vented Mask

CONTENTS: Innova® Hospital Nasal Non-Vented Mask with headgear  
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX

Rx ONLY MADE IN U.S.A.

The Innova® Hospital Nasal Non-Vented Mask is intended to be used as an accessory to CPAP/bi-level positive pressure systems containing exhalation valves. This product is indicated for use with CPAP/BI-LEVEL systems that have adequate alarms for positive pressure delivery failure. The mask is intended for SHORT TERM SINGLE PATIENT USE (maximum of 7 days) in the hospital or institutional environment on adult patients (>30kg/66 lbs).

#### GENERAL INFORMATION - WARNINGS AND CAUTIONS

**CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.

**CAUTION:** Do not expose the mask to temperatures above 140°F (60°C).

**CAUTION:** The mask is intended for SINGLE PATIENT USE ONLY. It is intended for short term use (not more than 7 days) and must not be used on multiple persons. Do not sterilize or disinfect.

**CAUTION:** As with all positive airway pressure systems, significant leak may occur between the mask and patient's face. The leak may vary depending on the patient's position and sleep stage. Variation in leak may significantly affect patient ventilation, potentially causing profound hypoventilation or hyperventilation. The magnitude of this effect depends on the positive pressure device's ability to compensate for leak. □

**WARNING:** This mask should not be used if the patient cannot respond to potential positive pressure device failure by breathing through their mouth.

**WARNING:** This mask should be used with a POSITIVE PRESSURE DEVICE WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure device is turned on and operating properly. When the device is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

**WARNING:** This mask must be used with devices which have ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a device failure. In the event of device failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

**WARNING:** If oxygen is used with this mask, the oxygen flow must be turned off when the positive pressure device is not operating. Explanation of the warning: When the positive pressure device is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.

**WARNING:** No smoking or open flames such as candles may be used when oxygen is in use.

**CAUTION:** Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

**CAUTION:** When used in combination with other medical devices not intended to be combined with the mask, safety can be decreased or performance can be altered (e.g. in combination with a humidifier for medical use, nebulizer, heat and moisture exchanger (HME), filters, bi-level positive airway pressure equipment, self-adjusting equipment, or additional oxygen supply or any exhaust port).

**WARNING:** Do not use this mask with baseline pressure less than 4 cm H<sub>2</sub>O.

**WARNING:** If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

**WARNING:** Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

**WARNING:** Significant unintentional leak may occur without properly fitting the mask. Follow the user instructions provided below.

**WARNING:** frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the Innova Nasal Non Vented Mask and consequently the safety or the quality of the therapy.

#### GENERAL INFORMATION

The mask has a service life of 7 days. Please dispose of the mask and accessories in accordance with local and regional guidelines. This mask and all its accessories do not contain natural rubber latex. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

**NOTE:** The mask without vent holes can only be used with positive pressure devices that have an exhaust valve.

#### CLEANING AND MAINTENANCE - SINGLE PATIENT USE

This mask is disposable and is intended for single patient short term use (maximum of 7 days) only. The mask should be discarded if heavily soiled. The mask may be cleaned daily or as needed using a 70% v/v isopropyl

alcohol swab or by wiping with a damp, soapy cloth. If using a damp cloth, use mild soap such as Ivory®. Rinse thoroughly and allow to air dry, out of direct sunlight.

**NOTE:** Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

**NOTE:** The gel cushion, although silky to the touch, will not tear, leak, or detach from the mask under normal use. The gel cushion is not indestructible. It will tear, cut, and rip if subjected to mishandling or abuse.

#### TECHNICAL INFORMATION

Therapy pressure range: 4-20 cmH<sub>2</sub>O

Resistance to air flow at 30 L/min: 0.10 cmH<sub>2</sub>O

Resistance to air flow at 50 L/min: 0.33 cmH<sub>2</sub>O

Resistance to air flow at 60 L/min: 0.46 cmH<sub>2</sub>O

Resistance to air flow at 100 L/min: 1.42 cmH<sub>2</sub>O

Dead space (approx): **S/M size** - 78 ml      **M/L size** - 100 ml

Operating Temperature: Do not expose the mask to temperatures above 140°F ( 60°C)

#### USER INSTRUCTIONS FOR THE INNOVA® NASAL MASK

1. Remove the mask and the headgear from the package.
  2. Place the headgear on a table, label side down with the narrowest part close to you.
  3. Set the mask with the gel side down onto the center of the headgear.
  4. Take each headgear strap and connect it to its respective location on the mask by looping the strap through the headgear slots and attaching it back to itself (Figure 1).
  5. Once the headgear has been properly attached to the mask, follow the fitting tips below to fit the mask:
  6. Detach one headgear strap by sliding the loop off the quick release hook (Figure 2), slip the headgear over the head and center the mask on the face.
  7. Find the detached headgear strap and slide the loop back on the quick release hook (Figure 2).
  8. Be sure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head (Figure 3).
  9. Adjust the bottom headgear straps by using the hook and loop tabs. Make sure the Active Headgear Connector™ is parallel to the face and that there is equal tension from the headgear on both sides. After tightening, make sure the Active Headgear Connector™ can move freely (Figure 4).
- NOTE:** Do NOT over tighten the headgear.
- NOTE:** The Touchless Spacebar™ will be tilted away from the face after tightening the bottom straps.
10. Adjust the upper headgear straps by using the tabs. Be sure they are tightened evenly and that the Touchless Spacebar™ is parallel to the forehead (Figure 5).
- NOTE:** The Touchless Spacebar™ should NOT touch the forehead. It should be parallel to the forehead (Figure 6).
11. Adjust the bottom headgear straps to make sure the bottom of the mask is sealed comfortably on the face.
  12. For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the hook and loop tabs, tightening or loosening the straps, and reattaching the tabs.
  13. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 7).
  14. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
  15. In the event of leaks, tighten the hook and loop tabs closest to the area of leak. Do NOT over tighten the headgear.

Innova® is a registered trademark of Sleepnet Corporation.

Visit our website at [www.sleepnetmasks.com](http://www.sleepnetmasks.com).

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Innova® es una marca comercial de Sleepnet Corporation.
**REZEPTPFLICHTIG GEBILDET IN U.S.A.**