**Explanation of Symbols
Explicación de símbolos
Erklärung der Symbole
Explication des symboles
Toelichting bij symbolen
Spiegazione dei simboli
Επεξήγηση των συμβόλων**

**Forklaring på symboler
Förklaring av symboler
Explicação dos símbolos
Wyjaśnienie symboli
Merkkien selitys
Sembollerin açıklaması
Symbolforklaring**

Caution or Warning
Precaución o advertencia
Achtung oder Warnung
Avertissement ou mise en garde
Attentiesignaal of waarschuwing
Attenzione o avvertenza
Επισημάνση προσοχής ή προειδοποίηση
Forsigtighed eller advarsel
Försiktighet eller varning
Cuidado ou advertência
Uwaga lub ostrzeżenie
Huomautus tai varoitus
Uyarı ve Önlem
Varsel eller advarsel

Lot number
Número de lote
Losnummer
Numéro du lot
Partijnummer
Numero di lotto
Αριθμός παρτίδας
Varenummer
Partinummer
Número do lote
Numer partii
Eränumero
Lot numarası
Partinummer

Manufacturer
Fabricante
Hersteller
Fabricant
Fabrikant
Fabbicante
Κατασκευαστής
Producent
Tilverkare
Fabricante
Producent
Valmistaja
Üretici
Fabrikant

Does not contain natural rubber latex
No contiene látex de caucho natural
Enthält keinen Naturkautschuklatex
Ne contient pas de latex en caoutchouc naturel
Bevat geen latex van natuurlijk rubber
Non contiene lattice di gomma naturale
Δεν περιέχει λάτεξ από φυσικό καουτσούκ
Inneholder ikke naturgummilætex
Innehåller inte rågummilætex
Não contém látex de borracha natural
Nie zawiera lateksu naturalnego.
Ei sisällä luonnonkumista valmistettua lateksia.
Doğal kauçuk lateks içermez.
Inneholder ikke naturlig gummilætex

Part number
Número de pieza
Teilenummer
Référence
Onderdeelnummer
Numero di articolo
Αριθμός εξαρτήματος
Del nummer
Delnummer
Número da peça
Numer części
Osanumero
Parça numarası
Delenummer

Medical Device
Producto sanitario
Medizinisches Gerät
Dispositif médical
Medisch hulpmiddel
Dispositivo medico
Ιατρική Συσκευή
Medicinsk utrustning
Medicinsk enhed
Dispositivo médico
Urządzenie medyczne
Lääkinlääkinnön laite
Medikal Cihaz
Medisinsk enhet

Not MRI Safe
No segura para resonancia magnética
Nicht MRT/MRI-geschützt
Ne pas être utilisé pendant une IRM
Non compatible con la risonanza magnetica
Niet bestand tegen MRI
Non compatibile con la risonanza magnetica
Όχι ασφαλές σε περιβάλλον μαγνητικού τομογράφου
Ikke MR-sikker
Får ej användas vid magnetröntgen
Não seguro para ressonância magnética
Nie nadaje się do bezpiecznego stosowania w środowisku rezonansu magnetycznego
Ei MRI-turvallinen
MRI için Uygun Değil
Ikke MRI-sikker

Use by Date
Fecha de caducidad
Verfallsdatum
Date limite d'utilisation
Uiterste houdbaarheidsdatum
Da utilizzare entro la data indicata
Χρησιμοποιήστε μέχρι την Ημερομηνία
Bör användas före
Holdbarhedsdato
Prazo de validade
Data przydatności do użycia
Viimeinen käyttöpäivä
Son Kullanma Tarihi
Brukes innen

Single use only
Un solo uso
Nur für einzigen Patienten verwenden
Usage unique seulement
Voor eenmalig gebruik
Eslusivamente monouso
Για μία μόνο χρήση
Kun enkelt brug
Endast för engångsbruk
Apenas para utilização única
Tylko do jednorazowego użytku
Vain yhden potilaan käyttöön
Tek kullannimik
Kun enkeltbruk

Medical Product Services GmbH
Borngasse 20, 35619 Braunfels Germany
info@mps-gmbh.eu

**User Instructions for
MiniMe® Pediatric Ventilation Mask and Headgear**

Instrucciones de usuario para la máscara pediátrica de ventilación y el casco MiniMe®
Gebrauchsanleitung für MiniMe® Pädiatrische Beatmungsмасke mit Kopfgurt
Mode d'emploi du casque de ventilation pédiatrique MiniMe®
Gebruiksaanwijzing voor het pediatische ventiliatiemaske en hoofdhulsel MiniMe®
Istruzioni per l'uso della cuffia e della Maschera pediatrica per ventilazione MiniMe®
Οδηγίες χρήσης Παιδιατρικής Μάσκας Αερισμού MiniMe® και Κεφαλοδέτη
Brugsanvisning til MiniMe® pædiatrisk ventilationsmaske og hovedtøj
Bruksanvisning för MiniMe® pediatrik ventilationsmask och maskhållare
Instruções de Utilização para a Máscara de Ventilação Pediátrica e Arnês MiniMe®
Instrukcja obsługi pediatrycznej kieszonkowej maski twarzowej MiniMe® i ramki na głowę

MiniMe® - lasten hengitysmaskin ja päitsien käyttöohje
MiniMe® Pediyatrik Ventilasyon Maskesi ve Kafa Bandı Kullanım Talimatları
Brukerveiledning for MiniMe® pediatrik ventilasjonsmaske og hodeutstyr

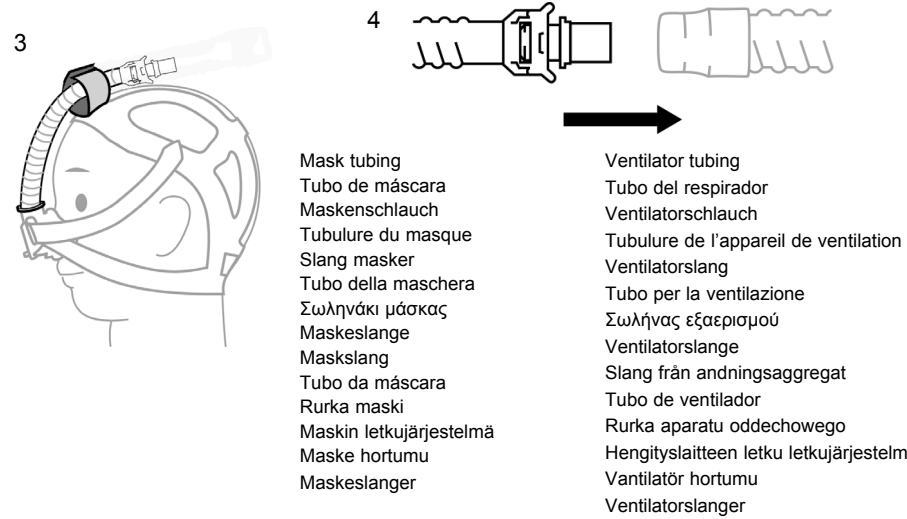
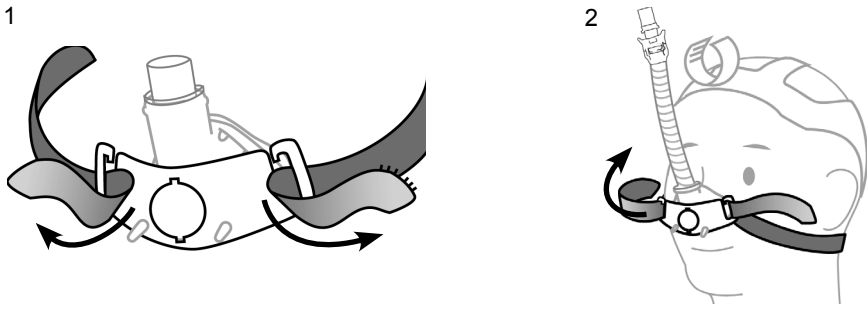
These instructions contain the following languages:
English, Spanish, German, French, Dutch, Italian, Greek,
Swedish, Danish, Portuguese, Polish, Finnish, Turkish, and
Norwegian

Sleepnet Corporation
5 Merrill Industrial Drive
Hampton, NH 03842 USA
Tel: (603) 758-6600
Fax: (603) 758-6699
Toll Free: 1-800-742-3646 (USA ONLY)
www.sleepnetmasks.com


90137 Rev. D 2021-07


- | | |
|-------------------------|---|
| Mask tubing | Ventilator tubing |
| Tubo de máscara | Tubo del respirador |
| Maskenschlauch | Ventilatorschlauch |
| Tubulure du masque | Tubulure de l'appareil de ventilation |
| Slang masker | Ventilatorslang |
| Tubo della maschera | Tubo per la ventilazione |
| Σωληνάκι μάσκας | Σωλήνας εξαερισμού |
| Maskeslange | Ventilatorslange |
| Maskslang | Slang från andningsaggregat |
| Tubo da máscara | Tubo de ventilador |
| Rurka maski | Rurka aparatu oddechowego |
| Maskin letkujärjestelmä | Hengityslaitteen letku letkujärjestelmä |
| Maske hortumu | Ventilatör hortumu |
| Maskeslanger | Ventilatorslanger |


This page intentionally left blank.

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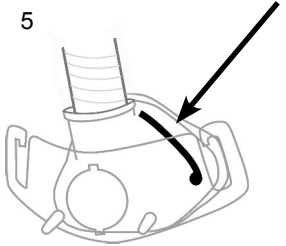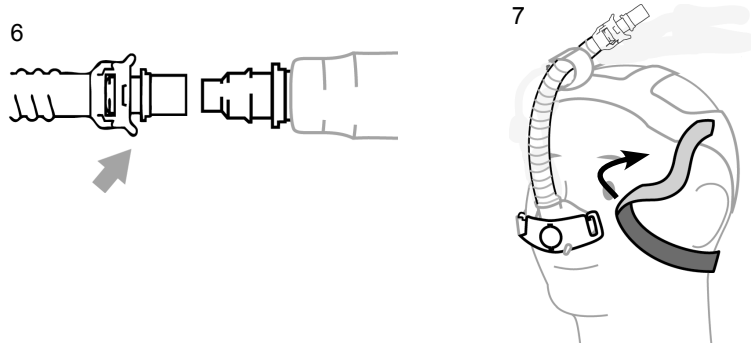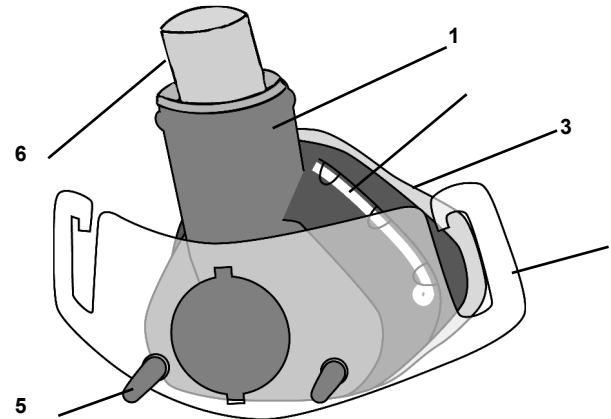This page intentionally left blank.

MiniMe® Pediatric Ventilation Mask

ENGLISH

CONTENTS: MiniMe® Pediatric Ventilation Mask, Tubing Assembly, and Headgear

Rx ONLY MADE IN U.S.A.

DOES NOT CONTAIN NATURAL RUBBER LATEX

The MiniMe® Pediatric Ventilation Mask is intended to provide a patient interface for application of noninvasive ventilation. The mask is to be used as an accessory to ventilators that have adequate alarms and safety systems for ventilator failure, and which are intended to administer positive pressure ventilation for treatment of respiratory failure or respiratory insufficiency. It is intended for patients >2 years old and <12 years old. The MiniMe® Pediatric Ventilation Mask is intended for single patient, multi-use in the home and in the hospital/institutional environment.

CAUTIONS AND WARNINGS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the MiniMe® Pediatric Ventilation Mask to temperatures above 122°F (50°C).

WARNING: This mask should be used with POSITIVE PRESSURE VENTILATOR WITH AN

EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the ventilator is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

WARNING: This mask must be used with ventilators which have ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

CAUTION: The MiniMe® Pediatric Ventilation Mask is designed to provide a stable, effective air seal with minimum headgear tensio.

Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. Due to the size of the mask, it is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: If oxygen is used with this mask, the oxygen flow must be turned off when the CPAP device is not operating. Explanation of the warning: when the ventilator is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the ventilator enclosure. Oxygen accumulated in the ventilator enclosure will create a risk of fire.

WARNING: No smoking or open flames (such as candles) may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

GENERAL INFORMATION

The MiniMe[®] Pediatric Mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com>. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: The MiniMe[®] Pediatric Mask and its accessories DO NOT CONTAIN NATURAL RUBBER LATEX.

NOTE: The MiniMe[®] Pediatric Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new MiniMe[®] Pediatric Mask with care.

NOTE: The MiniMe[®] Pediatric Mask without vent holes can only be used with ventilators having an active exhaust valve.

CLEANING AND MAINTENANCE

For Home Use: Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory or Dove. Rinse the mask thoroughly after washing. Visually verify that there are no obstructions and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.38 cm H₂O; at 100 L/min: 1.51 cm H₂O

DEADSPACE VOLUME (approx.) with tubing 57 mL, without tubing 21 mL.

User Instructions for MiniMe[®] Pediatric Mask Headgear

1. Place headgear on head, with blue fabric facing outward. Hook both straps on the open slots in the headgear connector (Figure 1).

2. Pull straps outward to center the mask over nose. Adjust straps. Headgear sizes Small and Medium feature additional Velcro[®] tabs (Figure 2 and 3).

NOTE: The mask should comfortably cover the nose, and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict the nose; child should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

3. Attach mask tubing to CPAP tubing; turn device on and verify airflow through mask (Figure 4).

4. If you notice any air leaks, gently press along the Memory Wire[™] to seal the leaks (Figure 5).

NOTE: If leaks persist despite customization of Memory Wire and adjustment of headgear straps, try moving the mask higher or lower around the nose to find the best fit. Velcro headgear straps can be readjusted to improve fit as well.

REMOVING THE MASK AND HEADGEAR

5. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing (Figure 6).

6. Open one of the horizontal straps and detach from headgear connector on the mask (Figure 7).

MiniMe[®] is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com

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