Explanation of Symbols Which Appear on the Device Labeling
 Explicación de símbolos que aparecen en la etiqueta del dispositivo
 Erklärung der Symbole auf dem Etikett dieses Gerätes
 Explication des symboles qui apparaissent sur l'étiquette de l'appareil
 Spiegazione dei simboli che figurano nelle etichette del dispositivo
 Пояснение символов, используемых для маркировки устройства

Caution or Warning
 Precaución o advertencia
 Achtung oder Warnung
 Avertissement ou mise en garde
 Attenzione o avvertenza
 Внимание или Осторожно

Refer to Instructions for Use
 Consulte las instrucciones de uso.
 Siehe Gebrauchsanleitung
 Se reporter au Mode d'emploi
 Consultare le Istruzioni per l'uso
 См. инструкции по применению

CE Certification Mark
 Marca de certificación CE
 CE Zertifizierungsmarke Marque CE
 Marcatura CE
 Знак сертификации CE

Lot number
 Número de lote
 Losnummer
 Numéro du lot
 Numero di lotto
 Номер партии

Manufacturer
 Fabricante
 Hersteller
 Fabricant
 Fabricante
 Изготовитель

Part number
 Número de pieza
 Teilenummer
 Référence
 Numero di articolo
 Номер по каталогу

This Product is not manufactured with Natural Rubber Latex.
 Este producto no se fabrica con látex de caucho.
 Dieses Produkt ist nicht mit Kautschuklatex hergestellt.
 Ce produit est fabriqué sans latex.
 Il prodotto non contiene lattice.
 Настоящий продукт изготавливается без применения натурального каучукового латекса.

Use by Date
 Fecha de caducidad
 Verfallsdatum
 Date limite d'utilisation
 Da utilizzare entro la data indicata
 Использовать до указанной даты

Date of Manufacture
 Fecha de fabricación
 Herstellungsdatum
 Date de fabrication
 Data di produzione
 Дата изготовления

Medical Device
 Producto sanitario
 Medizinisches Gerät
 Dispositif médical
 Dispositivo medico
 Медицинское устройство

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Australian Sponsor:
 Breathing and Medical.
 127 Bowden St, Meadowbank, NSW 2114
 www.breathingandmedical.com.au
 1800 335 333

Medical Product Services GmbH
 Borngasse 20, 35619 Braunfels Germany
 info@mps-gmbh.eu

iQ[®] Nasal Mask with User Instructions for iQ[®] 3 Point Headgear

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These instructions contain the following languages:

English, Spanish, German, French, Italian, and Russian

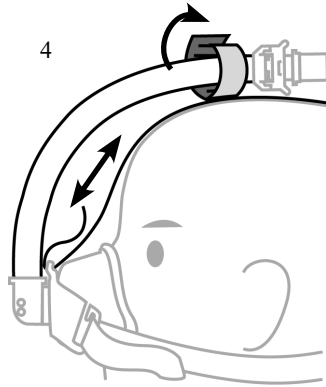
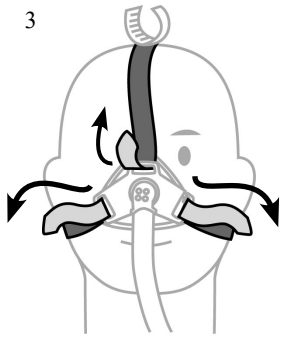
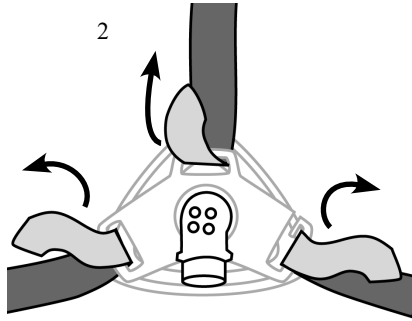
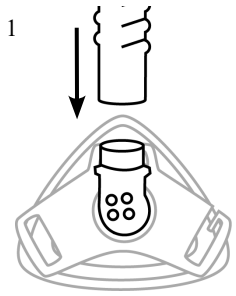
Sleepnet Corporation

5 Merrill Industrial Drive
 Hampton, NH 03842 USA
 Tel: (603) 758-6600
 Fax: (603) 758-6699
 Toll Free: 1-800-742-3646 (USA ONLY)
 www.sleepnetmasks.com

90112 Rev. M 2021-07

This page intentionally left blank.

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VENT HOLE PRESSURE/FLOW CHARACTERISTICS (Approx.)

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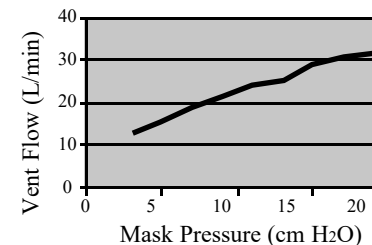
iQ® Nasal Mask

ENGLISH

CONTENTS: **iQ**® Nasal Mask, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation **iQ**® Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H₂O for the treatment of adult obstructive sleep apnea.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the **iQ**® Nasal Mask to temperatures above 122°F (50°C).

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Do not use the **iQ**® Nasal Mask with baseline pressure less than 3 cm H₂O or greater than 20 cm H₂O.

WARNING: Consult manufacturer for proper O₂ enrichment tubing.

GENERAL INFORMATION

The **iQ**® Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The **iQ**® Nasal Mask and its accessories are Latex Free. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new **iQ** Nasal Mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cm H₂O; at 100 L/min: 2.52 cm H₂O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

USER INSTRUCTIONS FOR **iQ** 3POINT HEADGEAR

1. Attach tubing to mask.
2. Thread lower straps through slots in the mask headgear connector, and thread center strap through top slot on headgear connector.
3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.
4. Adjust center strap. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
6. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
8. Open one of the horizontal straps and detach from headgear connector on the mask.

 **iQ** is a trademark of Sleepnet Corporation.
Visit our website at www.sleepnetmasks.com.

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**iQ**® ist ein Markenzeichen der Sleepnet Corporation.