Date of Manufacture / Fecha de fabricación /
Herstellungsdatum / Date de fabrication / Data
di produzione / Дата изготовления/ Verva
ardigingsdatum/ Ημερομηνία κατασκευής /
Fremstillingsdato / Tillverkningsdatum /
Data de fabrico / Data produkcyj /
Valmistuspäivämäärä / Üretim Tarihi / Дата на
производство / Datum proizvodnje / Datum
výroby / A gyártás dátuma / Produktionsdato /
Data fabricației / Datum proizvodnje / Datum
výroby / Datum izdelave

Use by Date / Fecha de caducidad /
Verfallsdatum / Date limite d'utilisation /
Da utilizzare entro la data indicata /
Использовать до указанной даты / Uiterste
houdbaarheidsdatum / Χρησιμοποιήστε
μέχρι την Ημερομηνία / Holdbarhedsdato /
Bör användas före / Prazo de validade /
Data przydatności do użycia / Viimeinen
käyttöpäivä / Son Kullanna Tarihi / Крайна
дата на използване / Upotrijebiti do datuma /
Datum použitelnosti / Felhasználhatósági
idő / Brukes innen / A se folosi până la
data / Rok upotrebe / Datum trvanlivosti /
Rok uporabnosti

Medical Device / Producto sanitario /Medizinisches Gerät / Dispositif médical /Dispositivo medico /
Медицинское устройство/ Medisch hulpmiddel / Ιατρική Συσκευή / Medicinsk enhed / Medicinsk
utrustning / Dispositivo médico / Urządzenie medyczne / Lääkinnällinen laite / Medikal Cihaz /
Медицинско изделие / Medicinski proizvod / Zdravotnický prostředek / Orvostechnikai eszköz /
Medisinsk enhet / Dispozitiv medical / Medicinsko sredstvo / Zdravotnícka pomôcka / Medicinski
priporoček

This Product is not manufactured with Natural Rubber Latex / Este producto no se fabrica con látex de
caucho / Dieses Produkt ist nicht mit Kautschuklatex hergestellt / Ce produit est fabriqué sans latex / Il
prodotto non contiene lattice / Настоящий продукт изготавливается без применения натурального
каучукового латекса./ Bevat geen latex van natuurlijk rubber / Δεν περιέχει λάτεξ από φυσικό
καουτσούκ / Ineholder ikke naturgummilætex / Innehåller inte rågummilætex / Não contém látex de
borracha natural / Nie zawiera lateksu naturalnego / Ei sisällä luonnonkumista valmistettua lateksia /
Doğal kauçuk lateks içermez / Тоzi продукт не е произведен с естествен каучуков латекс / Ovaj
proizvod nije proizveden od prirodnog lateksa / Tento výrobek není vyroben z přírodního latexu / Ez
a termék természetes gumi latex felhasználása nélkül készült / Dette produktet er ikke produsert med
naturlig gummilætex / Acest produs nu este fabricat cu latex din cauciuc natural / Ovaj proizvod nije
proizveden korišćenjem prirodne gume – lateksa / Tento výrobek nebol vyrobený z prírodného kaučuku /
Ta izdelek ni izdelan iz naravnega lateksa

Importer / Importador / Importeur / Importateur / Importeur / Импортёр / Importatore / Εισαγωγέας /
Importör / Importør / Importador / Importer / Maahantuaja / İthalatçı / Вноситель / Uvoznik / Importér /
Importör / Importør / Importator / Uvoznik / Dovoza / Uvoznik

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CONTENTS: iQ® Nasal Mask, Tubing Assembly, and Headgear
Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation iQ® Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H₂O for the treatment of adult obstructive sleep apnea.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the iQ® Nasal Mask to temperatures above 122°F (50°C).

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Do not use the iQ® Nasal Mask with baseline pressure less than 3 cm H₂O or greater than 20 cm H₂O.

WARNING: Consult manufacturer for proper O₂ enrichment tubing.

GENERAL INFORMATION

The iQ® Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The iQ® Nasal Mask and its accessories are Latex Free. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ® Nasal Mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cm H₂O; at 100 L/min: 2.52 cm H₂O

A-weighted Sound Pressure Level at 1 m distance: 29.9 dB (A); A-weighted Sound Power Level: 37.9 dB (A)

USER INSTRUCTIONS FOR IQ® 3POINT HEADGEAR

1. Attach tubing to mask.
2. Thread lower straps through slots in the mask headgear connector, and thread center strap through top slot on headgear connector.
3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.
4. Adjust center strap. Attach tubing to headgear with loop strap, if preferred.
NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.
5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
6. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.

8. Open one of the horizontal straps and detach from headgear connector on the mask.

iQ® is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com.

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iQ® é uma marca comercial da Sleepnet Corporation.