Explanation of Symbols
Explicación de símbolos
Erklärung der Symbole
Explication des symboles
Toelichting bij symbolen

Spiegazione dei simboli
Επεξήγηση των συμβόλων
Forklaring på symboler
Förklaring av symboler
Explicação dos símbolos

Wyjaśnienie symboli
Merkkien selitys
Sembollerin açıklaması
Пояснение символов

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CONTENTS: **Mojo®** Non Vented Full Face Mask with Headgear
Rx ONLY MADE IN U.S.A.
DOES NOT CONTAIN NATURAL RUBBER LATEX

INTENDED USE

The **Mojo®** Non Vented Full Face Mask is intended to provide a patient interface for application of non-invasive ventilation. The mask is to be used as an accessory to ventilators that have adequate alarms and safety systems for ventilator failure, and which are intended to administer positive pressure ventilation for treatment of respiratory failure or respiratory insufficiency. It is intended for use on adult patients (>30 kg.) who are appropriate candidates for noninvasive ventilation in the hospital or institutional environment.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.
CAUTION: Do not expose the **Mojo®** Non Vented Full Face Mask to temperatures above 122°F (50°C).
CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.
WARNING: This mask is not suitable for providing life support ventilation.
WARNING: This mask should be used with a POSITIVE PRESSURE VENTILATOR WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the ventilator is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.
WARNING: This mask must be used with ventilators which have adequate alarms and safety systems in the event of a ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.
WARNING: No smoking or open flames such as candles may be used when oxygen is in use.
WARNING: If using O₂ enrichment, turn off oxygen supply when not in use. When using a fixed flow rate of supplemental oxygen, the inhaled oxygen percentage will vary, depending on pressure settings, patient breathing pattern, mask size, and leak rate.
WARNING: The **Mojo®** Non Vented Full Face Mask must be used with non-conductive tubing.
WARNING: Do not use the **Mojo®** Non Vented Full Face Mask with baseline pressure less than 3 cm H₂O.
WARNING: To minimize the risk of vomiting during sleep, avoid eating or drinking for three (3) hours before using the mask.
WARNING: This product is indicated for use with POSITIVE PRESSURE VENTILATORS CONTAINING EXHALATION VALVES.
WARNING: This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed. This mask is contraindicated for patients without a spontaneous respiratory drive. This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask. This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia.
CAUTION: Consult physician prior to using mask if using any medications or devices to remove deep secretions.
WARNING: Any unusual chest discomfort, shortness of breath, gastric distention, abdominal pain, belching or flatulence from ingested air, or severe headache during or immediately after use should be immediately reported to physician.
WARNING: Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult a physician or dentist.
WARNING: Consult a physician if you experience any of the following symptoms while using the mask: drying of the eyes, eye pain, eye infections, or blurred vision. Consult an ophthalmologist if symptoms persist.
CAUTION: Any unusual skin irritation should be immediately reported to physician.
Do not use this product if you are experiencing nausea, vomiting, taking a prescription drug that may cause vomiting, or if you are unable to remove the mask on your own.
WARNING: Magnets are used in the mask and headgear clips. Ensure the mask and headgear clips are kept at a proper distance from any active medical implant to avoid possible effects from localized magnetic fields. The magnet will have a strength of less than 400mT at a distance of 2 inches.

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GENERAL INFORMATION

This mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit www.sleepnetmasks.com. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild shampoo or dish detergent. Rinse thoroughly after washing. Visually verify that the swivel elbow is not obstructed and allow the mask to air dry.

NOTE: Inspect mask and valve prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use. Follow recommended cleaning and maintenance protocols. The gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new **Mojo**® Non Vented Full Face Mask with care.

TECHNICAL INFORMATION

DEADSPACE VOLUME (mL approx.) **Small** 165 mL **Medium** 190 mL **Large** 210 mL
Extra Large 215 mL

USER INSTRUCTIONS FOR **MOJO**® HEADGEAR

1. Unhook the right or left magnetic clip from the mask.
2. Place the mask and headgear assembly on head with blue fabric facing outward. Hook the magnetic clip in place.
3. Adjust upper and lower headgear straps; unhook the Velcro® tabs and pull forward to center mask over face, and adjust strap tension and length. Affix tabs.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict your nose or mouth; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

4. Attach swivel adapter to device tubing (bi-level, ventilator); turn device on and verify airflow through mask.
5. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.
6. Rotate the tensioning screw to eliminate leaks at the bridge of the nose. Do not attempt to decrease bridge leaks by over-tightening the upper strap.

QUICK RELEASE OF MASK AND HEADGEAR

7. Grasp lower strap and lift away from face. Remove mask and headgear assembly.

Mojo® is a registered trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com

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Mojo® è un marchio di fabbrica di Sleepnet Corporation.