Date of Manufacture / Fecha de fabricación / Herstellungsdatum / Date de fabrication / Data di produzione / Дата изготовления / Vervaardigingsdatum / Ημερομηνία κατασκευής / Fremstillingsdato / Tillverkningsdatum / Data de fabrico / Data produkcji / Valmistuspäivämäärä / Uretim Tarihi / Дата на производство / Datum proizvodnje / Datum výroby / A gyártás dátuma / Produktionsdato / Data fabricației / Datum proizvodnje / Datum výroby / Datum izdelave

Use by Date / Fecha de caducidad / Verfallsdatum / Date limite d'utilisation / Da utilizzare entro la data indicata / Использовать до указанной даты / Uiterste houdbaarheidsdatum / Χρησιμοποιήστε μέχρι την Ημερομηνία / Holdbarhedsdato / Bőr användas före / Prazo de validade / Data przydatności do użycia / Viimeinen käyttöpäivä / Son Kullanna Tarihi / Крайняя дата на използване / Upotrijebiti do datuma / Datum použitelnosti / Felhasználhatósági idő / Brukes innen / A se folosi până la data / Rok upotrebe / Datum trvanlivosti / Rok uporabnosti

Medical Device / Producto sanitario / Medizinisches Gerät / Dispositif médical / Dispositivo medico / Медицинское устройство / Medisch hulpmiddel / Ιατρική Συσκευή / Medicinsk enhed / Medicinsk utrustning / Dispositivo médico / Urządzenie medyczne / Lääkinnällinen laite / Medikal Cihaz / Медицинско изделие / Medicinski proizvod / Zdravotnický prostředek / Orvostechnikai eszköz / Medicinsk enhet / Dispozitiv medical / Medicinsko sredstvo / Zdravotnícka pomôcka / Medicinski pripomoček

This Product is not manufactured with Natural Rubber Latex / Este producto no se fabrica con látex de caucho / Dieses Produkt ist nicht mit Kautschuklatex hergestellt / Ce produit est fabriqué sans latex / Il prodotto non contiene lattice / Настоящий продукт изготавливается без применения натурального каучукового латекса. / Bevat geen latex van natuurlijk rubber / Δεν περιέχει λάτεξ από φυσικό κροουτσούκ / Innehåller inte rågummilátex / Não contém látex de borracha natural / Nie zawiera lateksu naturalnego / Ei sisällä luonnonkumista valmistettua lateksia / Doğal kauçuk lateks içermez / Този продукт не е произведен с естествен каучуков латекс / Ovak proizvod nije proizveden od prirodnog lateksa / Tento výrobek není vyroben z přírodního latexu / Ez a termék természetes gumi latex felhasználása nélkül készült / Dette produktet er ikke produsert med naturlig gummilátex / Acest produs nu este fabricat cu latex din cauciuc natural / Ovak proizvod nije proizveden korišćenjem prirodne gume – lateksa / Tento výrobok nebol vyrobený z prírodného kaučuku / Та іздекек ни изделан из наравнега латекса

Importer / Importador / Importeur / Importateur / Importeur / Импортёр / Importatore / Εισαγωγέας / Importör / Importør / Importador / Importer / Maahantuaja / Ithalatçı / Вноситель / Uvoznik / Importér / Importör / Importør / Importator / Uvoznik / Dovoza / Uvoznik

Medical Product Services GmbH  
Borngasse 20, 35619 Braunfels Germany  
info@mps-gmbh.eu

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90131 Rev.K 2022-11

## iQ® Nasal Mask

### with User Instructions for iQ® StableFit Headgear

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#### These instructions contain the following languages:

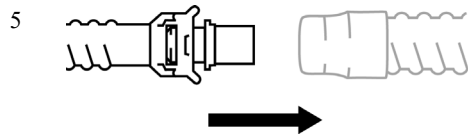
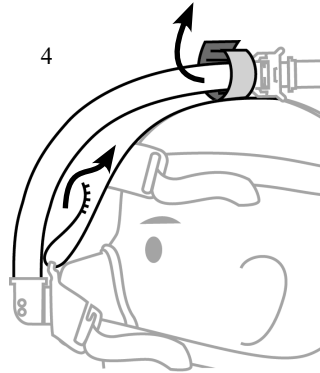
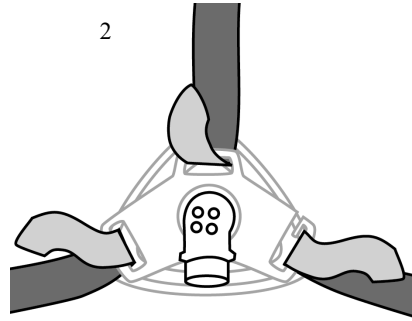
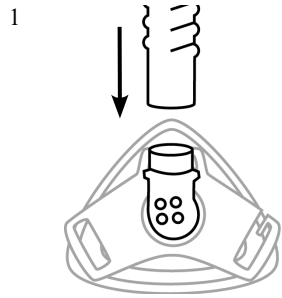
English, Spanish, German, French, Italian, Russian, Dutch, Greek, Danish, Swedish, Portuguese, Polish, Finnish, Turkish, Bulgarian, Croatian, Czech, Hungarian, Norwegian, Romanian, Serbian, Slovak, Slovenian

#### Sleepnet Corporation

5 Merrill Industrial Drive  
Hampton, NH 03842 USA  
Tel: (603) 758-6600  
Fax: (603) 758-6699  
Toll Free: 1-800-742-3646 (USA ONLY)  
www.sleepnetmasks.com

Explanation of Symbols / Explicación de símbolos / Erklärung der Symbole /  
 Explication des symboles / Spiegazione dei simboli / Пояснение символов /  
 Toelichting bij symbolen / Επεξήγηση των συμβόλων / Forklaring på symboler /  
 Förklaring av symboler / Explicação dos símbolos/ Wyjaśnienie symboli / Merkkien selitys /  
 Sembollerin açıklaması / Пояснение на символите / Objašnjenje simbola / Vysvětlení  
 symbolů / Jelmagyarázat / Symbolforklaring / Semnificația simbolurilor / Objašnjenje simbola /  
 Vysvetlenie symbolov / Razlaga simbolov

Mask tubing	Тръба за маска	CPAP tubing	CPAP тръба
Tubo de máscara	Cijevi maske	Tubo de CPAP	Cjevčice CPAP
Maskenschlauch	Hadička masky	CPAP-Schlauch	Hadička CPAP
Tubulure du masque	Maszk csővezés	Tubulure PPE	CPAP csővezés
Tubo della maschera	Maskeslange	Tubo del dispositivo CPAP	CPAP-slanger
Трубка маски	Tubulatura măștii	Трубка СИПАП	tubulatură CPAP
Slang van het masker	Crevo maske	CPAP-slang	Crevo CPAP aparata
Σωληνάκι της μάσκας	Hadička masky	Σωληνάκι CPAP	Hadička CPAP
Maskeslange	Cevi maske	CPAP-slange	Cev CPAP
Maskslang		CPAP-slang	
Tubo da máscara		Tubo PAPC	
Rurka maski		Rurka aparatu CPAP	
Naamarin letkujärjestelmä		CPAP-letkujärjestelmä	
Maske hortumu		CPAP hortumu	

Caution or Warning / Precaución o advertencia / Achtung oder Warnung / Avertissement ou mise en garde / Attenzione o avvertenza / Внимание или Осторожно / Attentionsignaal of waarschuwing / Επισημάνση προσοχής ή προειδοποίηση / Forsigtighed eller advarsel / Försiktighet eller varning / Cuidado ou advertência / Uwaga lub ostrzeżenie / Huomautus tai varoitus / Uyarı ve Önlem / Внимание или предупреждение / Opresz ili upozorenje / Upraznjenje / Óvatosságra intés vagy figyelmeztetés / Varsel eller advarsel / Precauții sau avertismente / Mera opreza ili upozorenje / Upozornenie alebo varovanie / Svarilo ali opozorilo

Refer to Instructions for Use / Consulte las instrucciones de uso / Siehe Gebrauchsanleitung / Se reporter au Mode d'emploi / Consultare le Istruzioni per l'uso / См. инструкции по применению / Raadpleeg de gebruiksaanwijzing / Ανατρέξτε στις Οδηγίες χρήσης / Der henvises til brugsanvisningen / Se bruksanvisningen / Consulte as instruções de uso / Patrz instrukcję obsługi / Katsko käyttöohjeet / Bkz. Kullanma Talimatları / Вижте инструкциите за употреба / Pogledajte upute za uporabu / Viz návod k použití / Lásd: Használati utasítás / Se bruksanvisningen / Consultati instrucțiunile de utilizare / Pogledati Uputstvo za upotrebu / Pozri pokyny na použitie / Glejte navodila za uporabo

CE Certification Mark / Marca de certificación CE / CE Zertifizierungsmarke / Marque CE / Marcatura CE / Знак сертификации CE / EG-keurmerk / Σήμα πιστοποίησης CE / CE certificeringsmærke / CE-certifieringsmärke / Marca de certificação CE / Oznaczenie certyfikacji CE / CE-sertifiointimerkintä / CE Onay İşareti / CE сертификационная маркировка / Certifikacijska oznaka CE / Označení CE certifikace / CE tanúsítvány-jelölés / CE-sertifiseringsmerke / Marcă de certificare CE / CE oznaka / Certificačná známka CE / Oznaka CE

Lot number / Número de lote / Losnummer / Numéro du lot / Numero di lotto / Номер партии / Partijnummer / Αριθμός παρτίδας / Varenummer / Partinummer / Número do lote / Numer partii / Eränumero / Номер на партида / Broj serije / Číslo šarže / Gyártási tételszám / Partinummer / Număr lot / Broj šarže / Číslo šarže / Številka serije

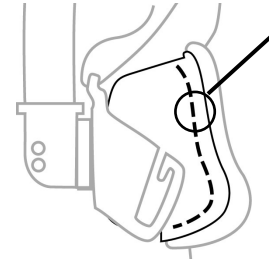
Manufacturer / Fabricante / Hersteller / Fabricant / Fabricante / Изготовитель / Fabrikant / Κατασκευαστής / Producent / Tillverkare / Fabricante / Producent / Valmistaja / Üretici / Производител / Proizvođač / Výrobce / Gyártó / Produsent / Producător / Proizvođač / Výrobca / Izdelovalec


Part number / Número de pieza / Teilenummer / Référence / Numero di articolo / Номер по каталогу / Onderdeelnummer / Αριθμός εξαρτήματος / Del nummer / Delnummer / Número da peça / Numer części / Osanumero / Parça numarası / Номер на част / Broj dijela / Číslo dílu / Cikkszám / Delenummer / Număr piesă / Serijski broj / Číslo dielu / Številka dela

6

Pliable ring	Muotoiltava rengas
Anillo flexible	Esnek halka
Biegsamer Ring	Пластичен пръстен
Anneau flexible	Savitljivi prsten
Anello pieghevole	Ohebný prstenec
Гибкое кольцо	НајлѠtható gyúró
Vuigbare ring	Bøyelig ring
Εύκαμπτος δακτύλιος	Inel pliable
Bøjelig ring	Savitljivi prsten
Formbar ring	Tvarovateľný prstenec
Anel flexível	Упогљив оброѠ
Miekkii piarscien	

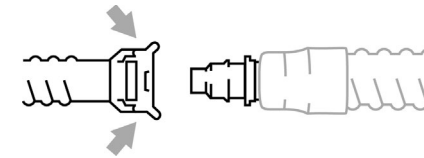
7


Squeeze	Purista
Presionar	Sıkıřtırma
Drücken	Стиснете
Presser	Istisnite
Premere	Stlaĉte
Нажмите	Nyomja össze
Indrukken	Klem
Πιέστε	Strángeři
Tryk	Stisnuti
Tryck	Stlaĉte
Apertar	Stisljivo
Ścisnąć	

This page intentionally left blank.

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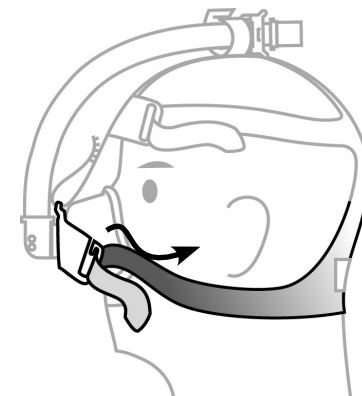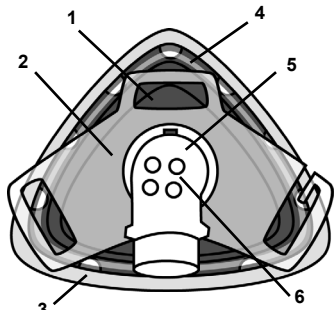CONTENTS: iQ® Nasal Mask, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation iQ® Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H<sub>2</sub>O for the treatment of adult obstructive sleep apnea.

#### GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the iQ® Nasal Mask to temperatures above 122°F (50°C).

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Do not use the iQ® Nasal Mask with baseline pressure less than 3 cm H<sub>2</sub>O or greater than 20 cm H<sub>2</sub>O.

WARNING: Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

#### GENERAL INFORMATION

The iQ® Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The iQ® Nasal Mask and its accessories are Latex Free. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

#### CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ® Nasal Mask with care.

#### TECHNICAL INFORMATION

Resistance to air flow at 50 l/min: 1.10 cm H<sub>2</sub>O; at 100 l/min: 2.52 cm H<sub>2</sub>O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

#### USER INSTRUCTIONS FOR IQ<sup>3</sup> STABLEFIT™ HEADGEAR

1. Attach tubing to mask.
2. Thread upper straps through forehead slots as illustrated, and affix Velcro® tabs lightly.
3. Thread lower straps through slots in the mask headgear connector. Thread center bridge strap through top slot on the headgear connector.
4. Place the mask and headgear assembly on head with blue fabric facing outward. Adjust upper and lower headgear straps; unhook the Velcro® tabs and pull forward to center mask over face, and adjust strap tension and length. Affix tabs.
5. Adjust center bridge strap. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

- Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
- Gently bend pliable ring embedded within mask to create a comfortable fit and reduce leaks.

#### REMOVING THE MASK AND HEADGEAR

- Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
- Open one of the horizontal straps and detach from headgear connector on the mask.

iQ® is a trademark of Sleepnet Corporation.

Visit our website at [www.sleepnetmasks.com](http://www.sleepnetmasks.com).

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iQ® é uma marca comercial da Sleepnet Corporation.