Innova® Hospital Nasal Vented Mask

Máscara nasal de ventilación para hospital Innova®
Innova® Krankenhaus Nasenmaske mit Auslassventil
Masque nasal ventilé Innova® pour milieu hospitalier
Maschera nasale ospedaliera Innova® ventilata
Innova® Hastane Nazal Havalandırma Maske
Innova® болнична назална маска с вентилация
Bolnička nazalna ventilirana maska Innova®
Zdravotnická nosní maska s větracími otvory Innova®
Innova® Ventilertet nasalmaske til hospitalsbrug
Innova®-ziekenhuisneusmasker voor non-invasieve beademing
Innova® ilma-aukollinen nenämaski sairaalakäyttöön
Νοσοκομειακή ρινική αεριζόμενη μάσκα Innova®
Innova® szellőző kórházi orrmask
Innova®病院用通気孔付きネーザルマスク
Innova® ventilertet nesemaske for sykehus
Szpitalna maska nosowa z otworem wydechowym Innova®
Máscara nasal hospitalar com ventilação Innova®
Mască nazală cu ventilație Innova® pentru uz spitalicesc
Вентилируемая назальная маска для больничного применения Innova®
Innova® bolnička nazalna maska sa ventilacijom
Nemocničná nosová ventilovaná maska Innova®
Bolnišnična ventilirana nosna maska Innova®
Innova® ventilertet näsmask för sjukhusbruk
Innova® كمامة أنفية مُهَوَّاة للمشفى

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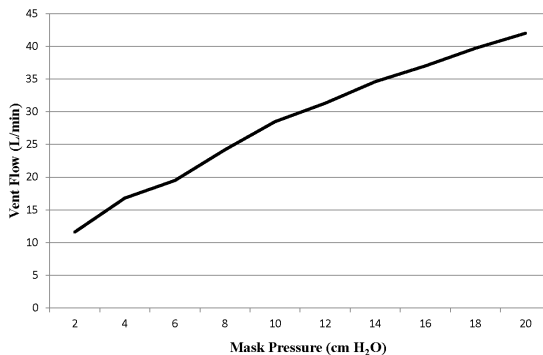
VENT HOLE PRESSURE/FLOW CHARACTERISTICS (Approx.)

CARACTERÍSTICAS DE PRESIÓN/FLUJO DEL ORIFICIO DE VENTILACIÓN (aprox.)
ENTLÜFTUNGSLÖCHDRUCK / FLIESSVERHALTEN (ungefähr)
CARACTÉRISTIQUES DE PRESSION/DÉBIT DES ORIFICES D'ÉVACUATION (approx.)
CARATTERISTICHE PRESSIONE/FLUSSO FORO DI SFOGO (Appross.)
HAVALANDIRMA DELİĞİ BASINÇ/AKIŞ ÖZELLİKLERİ (Yaklaşık olarak)
ΧΑΡΑΚΤΗΡΙΣΤΙΚΗ ΤΗΣ ΠΙΕΣΗΣ/ΡΟΗΣ ΟΠΙΣ ΕΞΑΕΡΙΣΜΟΥ (Κατά προσέγγιση)
ZNAČAJKE TLAKA/PROTOKA VENTILACIJSKOG OTVORA (približno)
CHARAKTERISTIKA TLAKU/PRŮTOKU VĚTRACÍMI OTVORY (přibliž.)
TRYK I VENTILERINGSKAPEN VENTILATIEOPENING (bij benadering)
ILMA-AUKON PAINE / VIRTAAUKON VENTILAUDET (likimäär.)
ΧΑΡΑΚΤΗΡΙΣΤΙΚΗ ΤΗΣ ΠΙΕΣΗΣ/ΡΟΗΣ ΟΠΙΣ ΕΞΑΕΡΙΣΜΟΥ (Κατά προσέγγιση)
SZELLŐZŐNYÍLÁS NYOMÁS/ÁRAMLÁS JELLEMZŐK (körülbelül)
通気孔圧力／流量特性 (近似値)
TRYKK-/STRØMNINGSKARAKTERISTIKA (omtrentlig) FOR VENTILASJONSHULL
CIŚNIENIE W OTWORZE WYDECHOWYM/CHARAKTERYSTYKA PRZEPŁYWU (w przybliżeniu)
CARACTERÍSTICAS DE PRESSÃO/FLUXO DO ORIFÍCIO DE VENTILAÇÃO (aprox.)
CARACTERISTICI PRIVIND PRESIUNEA/FLUXUL ORIFICIULUI DE VENTILAȚIE (Aprox.)
ΧΑΡΑΚΤΗΡΙΣΤΙΚΗ ΤΗΣ ΠΙΕΣΗΣ/ΡΟΗΣ ΟΠΙΣ ΕΞΑΕΡΙΣΜΟΥ (Κατά προσέγγιση)
KARAKTERISTIKE PRITISKA/PROTOKA OTVORA (približno)
CHARAKTERISTIKY TLAKU/PRIETOKU VENTILAČNÉHO OTVORU (približ.)
TLAK V ZRAČNIH ODPRTINAH/ZNAČILNOSTI PRETOKA (pribl.)
TRYCK-/FLÖDESEGENSKAPER FÖR VENTILATIONSÅL (ungefärligt)

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CONTENTS: Innova® Hospital Nasal Vented Mask with headgear
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX
Rx ONLY MADE IN U.S.A.

The Innova® Hospital Nasal Vented Mask is intended to provide an interface for application of positive airway pressure therapy, such as CPAP or bi-level. The mask is intended for SHORT TERM SINGLE PATIENT USE (maximum 7 days) on adult patients (>66lbs/30kg) in the hospital or institutional environment for whom CPAP or bi-level therapy has been prescribed.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the mask to temperatures above 140°F (60°C).

CAUTION: The mask is intended for SINGLE PATIENT USE ONLY. It is intended for short term use (not more than 7 days) and must not be used on multiple persons. Do not sterilize or disinfect.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Do not use this mask with baseline pressure less than 4 cm H₂O.

WARNING: Consult manufacturer for proper O₂ enrichment tubing.

WARNING: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

WARNING: Significant unintentional leak may occur without properly fitting the mask. Follow the USER INSTRUCTIONS FOR THE INNOVA® NASAL MASK provided.

GENERAL INFORMATION

The mask has a service life of 7 days. Please dispose of the mask and accessories in accordance with local and regional guidelines. This mask and all its accessories do not contain natural rubber latex. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

This mask is disposable and is intended for single patient short term use (maximum of 7 days) only. The mask should be discarded if heavily soiled. The mask may be cleaned daily or as needed using a 70% v/v isopropyl alcohol swab or by wiping with a damp, soapy cloth. If using a damp cloth, use mild soap. Rinse thoroughly and allow to air dry, out of direct sunlight.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to the touch, will not tear, leak, or detach from the mask under normal use. The gel cushion is not indestructible. It will tear, cut, and rip if subjected to mishandling or abuse.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.25 cmH₂O

Resistance to air flow at 100 L/min: 0.95 cmH₂O

Dead space (approx): **S/M size** - 76 ml **M/L size** - 102 ml

Operating Temperature: Do not expose the mask to temperatures above 140°F (60°C)

USER INTRUCTIONS FOR THE INNOVA[®] NASAL MASK

1. Remove the mask and the headgear from the package.
 2. Place the headgear on a table, label side down with the narrowest part close to you.
 3. Set the mask with the gel side down onto the center of the headgear.
 4. Take each headgear strap and connect it to its respective location on the mask by looping the Velcro strap through the headgear slots and attaching it back to itself (Figure 1).
 5. Once the headgear has been properly attached to the mask, follow the fitting tips below to fit the mask:
 6. Detach one Velcro strap by sliding the Velcro[®] loop off the quick release hook, slip the headgear over the head and center the mask on the face.
 7. Find the detached Velcro[®] strap and slide the Velcro loop back on the quick release hook (Figure 2).
 8. Be sure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head (Figure 3).
 9. Adjust the bottom headgear straps by using the Velcro[®] tabs. Make sure the Active Headgear Connector[™] is parallel to the face and that there is equal tension from the headgear on both sides. After tightening, make sure the Active Headgear Connector[™] can move freely (Figure 4).
- NOTE:** Do NOT over tighten the headgear.
- NOTE:** The Touchless Spacebar[™] will be tilted away from the face after tightening the bottom straps.
10. Adjust the upper headgear straps by using the tabs. Be sure they are tightened evenly and that the Touchless Spacebar[™] is parallel to the forehead (Figure 5).
- NOTE:** The Touchless Spacebar[™] should NOT touch the forehead. It should be parallel to the forehead (Figure 6).
11. Adjust the bottom headgear straps to make sure the bottom of the mask is sealed comfortably on the face.
 12. For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the Velcro[®] straps, tightening or loosening the straps, and reattaching the Velcro[®].
 13. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 7).
 14. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
 15. In the event of leaks, tighten the Velcro[®] tabs closest to the area of leak. Do NOT over tighten the headgear.

Innova[®] is a registered trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com.

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Medical Product Services GmbH