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1. Zanka za trak
2. Prednji vezni trak
3. Vodoravni trak
4. Okvir

# iQ<sup>®</sup> Ventilation Nasal Mask

CONTENTS: iQ<sup>®</sup> Ventilation Nasal Mask with iQ 3 Point Headgear  
Rx ONLY LATEX FREE MADE IN U.S.A.

## INTENDED USE

Use of this product is indicated for use with POSITIVE PRESSURE VENTILATORS CONTAINING EXHALATION VALVES. It is intended for use on adult patients.

## GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: This device is for sale by or on the order of a physician.

CAUTION: Do not expose the iQ<sup>®</sup> Ventilation Nasal Mask to temperatures above 122°F (50°C).

CAUTION: Do not sterilize or disinfect.

CAUTION: Single patient use only. May be cleaned and used repeatedly on the same person but must not be used on multiple persons.

WARNING: This mask should be used with a POSITIVE PRESSURE VENTILATOR WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. However, when the ventilator is not operating, exhaled air may be rebreathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

CAUTION: Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

WARNING: The iQ<sup>®</sup> Ventilation Nasal Mask must be used with non-conductive tubing.

## GENERAL INFORMATION

The iQ<sup>®</sup> Ventilation Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. Please dispose of the mask and accessories in accordance with local and regional guidelines. The iQ<sup>®</sup> Ventilation Nasal Mask and its accessories are Latex Free. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

## CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild shampoo or dish detergent. Rinse thoroughly after washing. Visually verify that the swivel elbow is not obstructed and allow the mask to air dry.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tears or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ<sup>®</sup> Ventilation Nasal Mask with care.

## USER INSTRUCTIONS FOR IQ<sup>®</sup> 3 POINT HEADGEAR

1. Attach tubing to mask, if not already assembled.
2. Thread lower straps through slots in the mask headgear connector, and thread center strap through top slot on headgear connector.
3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.

4. Adjust center strap. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal.

Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.

6. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

## REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
8. Open one of the horizontal straps and detach from headgear connector on the mask.

iQ<sup>®</sup> is a trademark of Sleepnet Corporation.

Visit our website at [www.sleepnetmasks.com](http://www.sleepnetmasks.com).

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Medical Product Services GmbH