User Instructions for MiniMe® Pediatric Mask and Headgear

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These instructions contain the following languages:

English, Spanish, French and Portuguese

Sleepnet Corporation

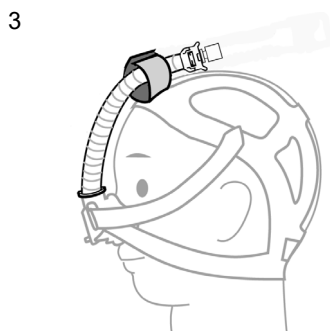
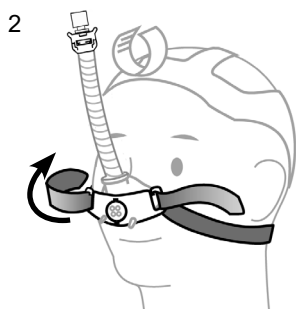
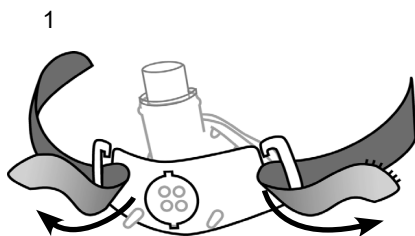
5 Merrill Industrial Drive
Hampton, NH 03842 USA

Tel: (603) 758-6600

Fax: (603) 758-6699


Toll Free: 1-800-742-3646 (USA ONLY)

www.sleepnetmasks.com




Mask tubing
Tubo de máscara
Tubulure du masque
Tubo da máscara

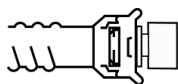4

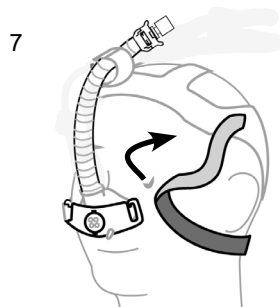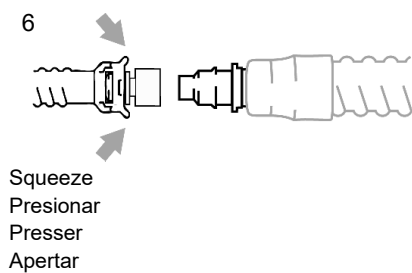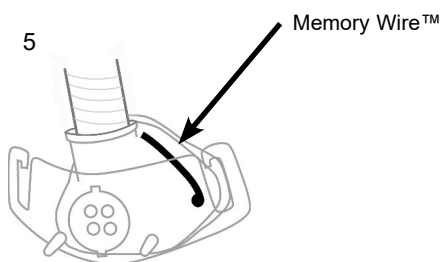CPAP tubing
Tubo de CPAP
Tubulure de l'appareil de
pression positive expiratoire
Tubo CPAP


VENT HOLE PRESSURE/FLOW CHARACTERISTICS (Approx.)

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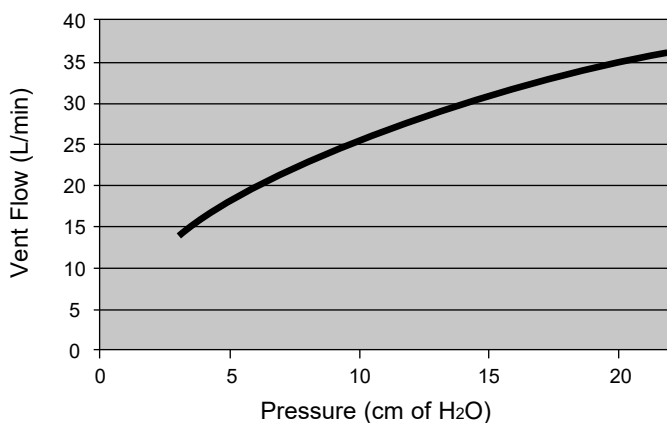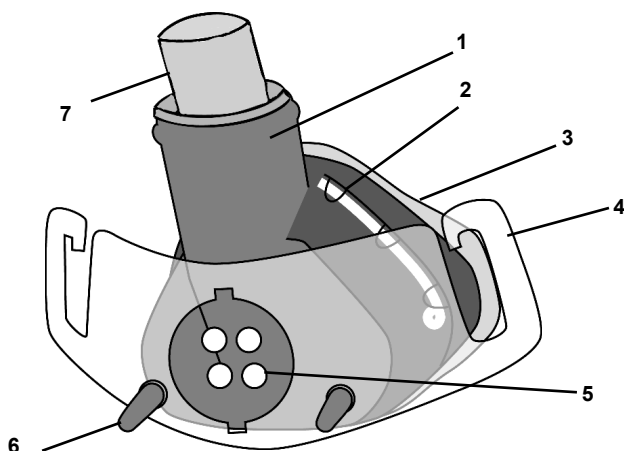
MiniMe® Pediatric Mask

CONTENTS: MiniMe® Pediatric Mask, Tubing Assembly, and Headgear

Rx ONLY MADE IN U.S.A.

DOES NOT CONTAIN NATURAL RUBBER LATEX

The MiniMe® Pediatric Mask is intended to provide an interface for application of CPAP or bi-level therapy. It is intended for patients >2 years old and <12 years old. The MiniMe® Pediatric Mask is intended for single patient, multi-use in the home and in the hospital/institutional environment.

CAUTIONS AND WARNINGS

- CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.
- CAUTION:** Do not expose the MiniMe® Pediatric Mask to temperatures above 122°F (50°C).
- CAUTION:** Do not use the MiniMe® Pediatric Mask with baseline pressure less than 3 cm H₂O or greater than 20 cm H₂O.
- CAUTION:** The MiniMe® Pediatric Mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.
- CAUTION:** If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.
- WARNING:** Do not block or try to seal the vent holes. (If using the non-vented style mask, you must use with a patient circuit that incorporates an exhalation valve as part of the circuit assembly.)
- WARNING:** At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- WARNING:** This mask is intended for use on children >2 years of age and <12 years of age. Due to the size of the mask, it is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.
- WARNING:** Children should be attended at all times during therapy.
- WARNING:** This mask should be used with the positive airway pressure device recommended by the manufacturer, or by your physician or respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be rebreathed. Rebreathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of positive airway pressure devices and masks.
- WARNING:** If oxygen is used with this mask, the oxygen flow must be turned off when the CPAP device is not operating. Explanation of the warning: when the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the CPAP device enclosure. Oxygen accumulated in the CPAP device enclosure will create a risk of fire.
- WARNING:** No smoking or open flames (such as candles) may be used when oxygen is in use.
- WARNING:** Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

GENERAL INFORMATION

The MiniMe® Pediatric Mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com/>.

NOTE: The mask has a service life of 6 months. **NOTE:** Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: The MiniMe® Pediatric Mask and its accessories DO NOT CONTAIN NATURAL RUBBER LATEX.

NOTE: The MiniMe® Pediatric Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new MiniMe® Pediatric Mask with care.

NOTE: The MiniMe® Pediatric Mask without vent holes can only be used with ventilators having an active exhaust valve.

CLEANING AND MAINTENANCE

For Home Use: Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory or Dove. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask and vent holes prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures. Vent holes should not be blocked.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.38 cm H₂O; at 100 L/min: 1.51 cm H₂O

DEADSPACE VOLUME (approx.) with tubing 57 mL, without tubing 21 mL.

User Instructions for MiniMe® Pediatric Mask Headgear

1. Place headgear on head, with blue fabric facing outward. Hook both straps on the open slots in the headgear connector (Figure 1).
2. Pull straps outward to center the mask over nose. Adjust straps. Headgear sizes Small and Medium feature additional Velcro® tabs (Figure 2 and 3).

NOTE: The mask should comfortably cover the nose, and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict the nose; child should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

3. Attach mask tubing to CPAP tubing; turn device on and verify airflow through mask (Figure 4).
4. If you notice any air leaks, gently press along the Memory Wire™ to seal the leaks (Figure 5).

NOTE: If leaks persist despite customization of Memory Wire and adjustment of headgear straps, try moving the mask higher or lower around the nose to find the best fit. Velcro headgear straps can be readjusted to improve fit as well.

REMOVING THE MASK AND HEADGEAR

5. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing (Figure 6).
6. Open one of the horizontal straps and detach from headgear connector on the mask (Figure 7).

MiniMe® is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com

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